

AKR 550



USER MANUAL


CONTENTS

I. INTRODUCTION	4
II. SAFETY CONSIDERATION	7
1. General cautions	8
2. Precautions regarding IT network	8
3. Electromagnetic compatibility	8
III. ACCESSORIES	12
IV. DEVICE	14
1. General description of product	15
2. Intended use defined	15
3. Classification defined, rule given	15
4. Classification of device	15
5. Usage of product	15
6. Mode of operation	15
7. Parts identification	16
V. INSTRUCTIONS FOR USE	17
1. Conveyance	18
2. Installation	18
3. Connection / Wiring	18
4. Maintenance / Inspection	19
5. Disposal	19
VI. HOW TO USE	20
1. Operation procedure	21
2. Measurement flow	21
3. Measurement	22
a. Preparation for measurement	22
b. Power distribution	22
c. Standby	23
d. Preparation of examinee	23
e. Alignment	24
f. Measurement	26
g. Print-out of measurement result	27
4. Setting of [Setup] screen	30
a. [Number]	32
b. [Language]	32
c. [Customize]	33
d. [Date form]	34
e. [Message]	34
f. [Default setting]	35
5. Scotopic Pupil Size (SPS) - measurement function	35
6. IOL measurement function	36
7. Low reliability mark display function	37
8. Output	37
9. Data screen function	38
10. Power saving function	40
11. Contact lens: measurement of base curve	40

VII. STORAGE AND MAINTENANCE	41
1. Reload of printer paper	42
2. Fuse replacement	42
3. Setting of chinrest liner	43
4. Storage of device	43
5. Confirmation of measurement accuracy	44
6. Periodical inspection and maintenance	44
VIII. TIPS FOR EFFECTIVE MEASUREMENT	46
IX. ERROR DISPLAY	48
X. TROUBLESHOOTING	50
XI. SPECIFICATIONS	52
XII. QR CODE	54

I. INTRODUCTION



 The complete user manual is available on a web space.
 To access other available languages, please scan the QR code at the end of this user manual > Chapter QR code (p.54).












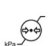












Please read this manual thoroughly so that effective operation is ensured.

1. The information contained in this manual is subject to change without notice.
2. While reasonable efforts have been made in the preparation of this document to ensure its accuracy, you should contact your local distributor immediately if any quarries arise due to editorial errors or omissions etc.
3. If you find any imperfect collating or missing pages, contact your local distributor for replacement.




This manual includes important contents to prevent users or the others from harms and to use this device safely. This device (AKR550) can objectively measure the refractive power of the eye.

Read this manual after understanding the symbols below and follow the instructions in use.


Symbols

	This symbol indicates that mishandling as a result of failure to comply with the indications can result in "personal death or serious injury"
	Denotes general ban or prohibition
	General mandatory action
	Additional information which is important to the text or useful/convenient to know
	Refer to operation manual
	Do not reuse
	Serial No.
	Catalog number
	ON/OFF switch (mean of isolation from the supply means)
	Number on the left is the lower limit and the one on the right is the upper limit of the temperature
	Number on the left is the lower limit and the one on the right is the upper limit of the humidity
	The number on the left is the lower limit and the one on the right is the upper limit of the atmospheric pressure
	EU Battery Directive
	WEEE symbol
	Avoid direct sunlight
	This is the type B equipment
	Manufacturing date (year)
	Symbol for "manufacturer"
	Symbol for Compliant with CE marking i.e. with applicable European directives
	This way up
	Fragile
	Keep dry
	Stacking limit by number
	Medical device

Disposal

- | | |
|---|--|
|  | When it reaches the end of its lifetime, the instrument should not be thrown out with the household refuse. It can be disposed of at a waste management center operated by the municipality or the retailers who offer this service. |
|  | This manual includes the information about basic operation, inspection and maintenance etc. of AKR550. This device and the contents of this manual comply with IEC60601-1. |
|  | The current version of the product software is V1.0.1. |

II. SAFETY CONSIDERATION

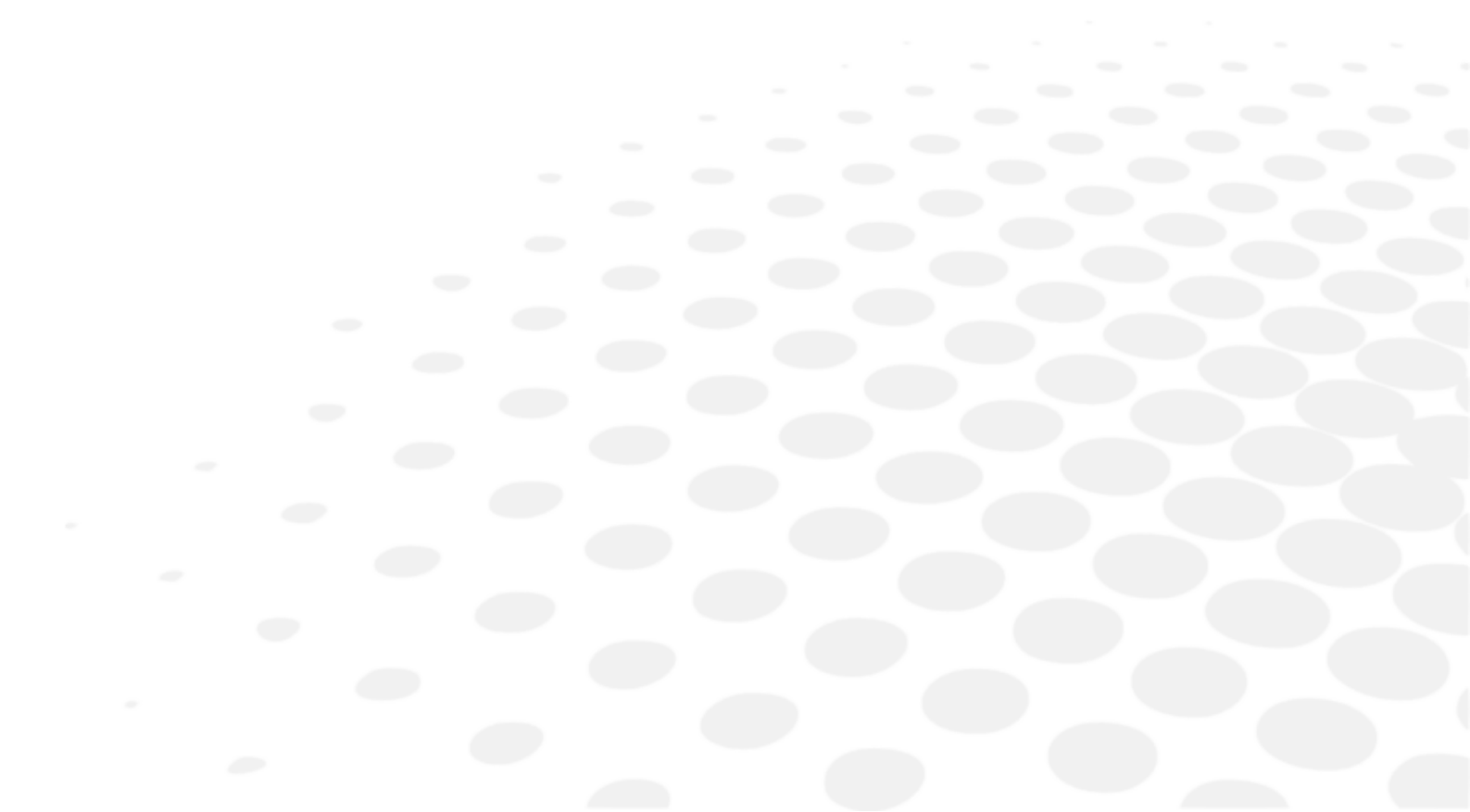

1. General cautions

- It affects its measurement accuracy if fingerprints or dust etc. are on the optical parts, such as the lens of the view window. Do not touch them with hands and avoid dust.
- If fingerprints or dust are adhered on the optical parts, such as lens, wipe it gently with a soft cloth.
- Observe the following environmental conditions for use and storage and transportation.
- The device is not intended to be used in conjunction with oxygen rich environment.
- Never attempt to modify or disassemble this product yourself. It can result in malfunction or fire.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

	Temperature	Humidity	Atmospheric pressure
Use	[10°C ; 40°C]	[30% ; 90%]	[800hPa ; 1060hPa]
Storage	[-10°C ; 55°C]	[10% ; 95%]	[700hPa ; 1060hPa]
Transportation	[-40°C ; 70°C]	[10% ; 95%]	[500hPa ; 1060hPa]

- Avoid installation near TV or radio. The reception can be disturbed by electrical noise.
- If liquid is spilled on this device or a foreign substance is entered in it, unplug the power cord and contact your local distributor.
- Cut the power immediately and contact your local distributor if malfunction (noise, smoke etc.) occurs. It can result in fire or injury if you keep using it.
- If malfunction occurs, do not touch the inside of this device. Unplug the power cord and contact your local distributor.
- No contraindication.

2. Precautions regarding IT network

- This device can output the data to PC and so on through RS232C interface.
- Connection of this device to an IT network that includes other equipment could result in previously unidentified risks to patients, operators or third parties.
- The responsible organization should identify, analyze, evaluate and control these risks.
- Subsequent changes to the IT network could introduce new risks and require additional analysis.
- Changes to the IT network include:
 - Changes in the IT network configuration
 - Connection of additional items to the IT network
 - Disconnecting items from the IT network
 - Update of equipment connected to the IT network, and
 - Upgrade of equipment connected to the IT network
- Please contact your distributor about the detail of this device.

3. Electromagnetic compatibility

AKR550 conforms to the requirements of the EMC (electromagnetic compatibility) standard.

1. This product needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in this manual.
2. Portable and mobile RF communications equipment can affect medical electrical equipment.
3. The use of accessories, transducers and cables other than those specified, with the exception of transducers and cables sold by the manufacturer of the equipment or system as replacement parts for internal components, may result in increased emissions or decreased immunity of the equipment or system.
4. The equipment or system should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.

5. The use of the accessory, transducer or cable with equipment and systems other than those specified may result in increased emission or decreased immunity of the equipment or system.

Phenomenon	Home healthcare environment	Compliance
Conducted and radiated RF emissions	CISPR 11	Class A, Group 1
Harmonic distortion	IEC 61000-3-2	Class A
Voltage fluctuations and flicker	IEC 61000-3-3	Complies

It does not intend to use in aircraft and vehicle.

The emission characteristic of AKR550 is for use at industrial area and hospitals (CISPR11 Class A). In case of using this device at domiciliary environment (in this case CISPR11 Class B is required), this device might not be able to provide sufficient protection to the radio-frequency communication service.

The user might need to take countermeasures such as rearrangement or reorientation of the device.

Phenomenon	Basic EMC standard or test method	Immunity test levels Home healthcare environment	Compliance level
Electrostatic discharge	IEC 61000-4-2	± 8kV contact ± 15kV air	± 8kV contact ± 15kV air
Radiated RF EM fields	IEC 61000-4-3	10 V/m ^a 80 MHz to 2.7 GHz 80% AM at 1 kHz	10V/m
Proximity fields from RF wireless communications equipment		See the table below.	
Rated power frequency magnetic fields	IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m

^a Before modulation is applied.


Test frequency (MHz)	Band ^a (MHz)	Service ^a	Modulation ^b	Maximum power (W)	Distance (m)	Immunity test level (V/m)	Compliance level
385	380 - 390	TETRA400	Pulse modulation ^b 18Hz	1.8	0.3	27	27
450	430 - 470	GMRS460, FRS460	FM ±5kHz deviation 1kHz sine	2	0.3	28	28
710	704 - 787	LTE Band 13, 17	Pulse modulation ^b 217Hz	0.2	0.3	9	9
745							
780							
810	800 - 960	GSM800/900, TETRA800, iDEN820, CDMA850, LTE Band 5	Pulse modulation ^b 18Hz	2	0.3	28	28
870							
930							
1720	1700 - 1990	GSM1800 ; CDMA1900 ; GSM1900 ; DECT ; LTE Band 1, 3, 4, 25 ; UMTS	Pulse modulation ^b 217Hz	2	0.3	28	28
1845							
1790							
2450	2400 - 2570	Bluetooth, WLAN, 802.11b/g/n, FRID2450, LTE Band 7	Pulse modulation ^b 217Hz	2	0.3	28	28
5240	5100 - 5800	WLAN 802.11a/n	Pulse modulation ^b 217Hz	0.2	0.3	9	9
5500							
5785							




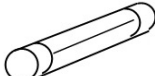
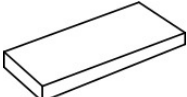



^a For some services, only the uplink frequencies are included.

^b The carrier shall be modulated using a 50% duty cycle square wave signal.


Phenomenon	Basic EMC standard	Immunity test levels Home healthcare environment	Compliance level
Electrical fast transients/ bursts	IEC 61000-4-4	Input AC power port ± 2kV 100 kHz repetition frequency	± 2kV
		Signal input/ output unit port ±1kV 100 kHz repetition frequency	± 1kV
Surges Line-to-line	IEC 61000-4-5	± 1kV	
Surges Line-to-ground		± 2kV	
Conducted disturbances induced by RF fields	IEC 61000-4-6	3 Vrms 0.15 MHz - 80 MHz 6 Vrms in ISM bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	3 Vrms
Voltage dips	IEC 61000-4-11	0% U _T ; 0.5 cycle 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	0% U _T ; 0.5 cycle
		0% U _T ; 1 cycle and 70% U _T ; 25 cycle Single phase: 0°	0% U _T ; 1 cycle 70% U _T ; 25 cycle
Voltage interruptions		0% U _T ; 250 cycle	0%U _T ; 250 cycle
U _T is the AC supply voltage before applying inspection level.			

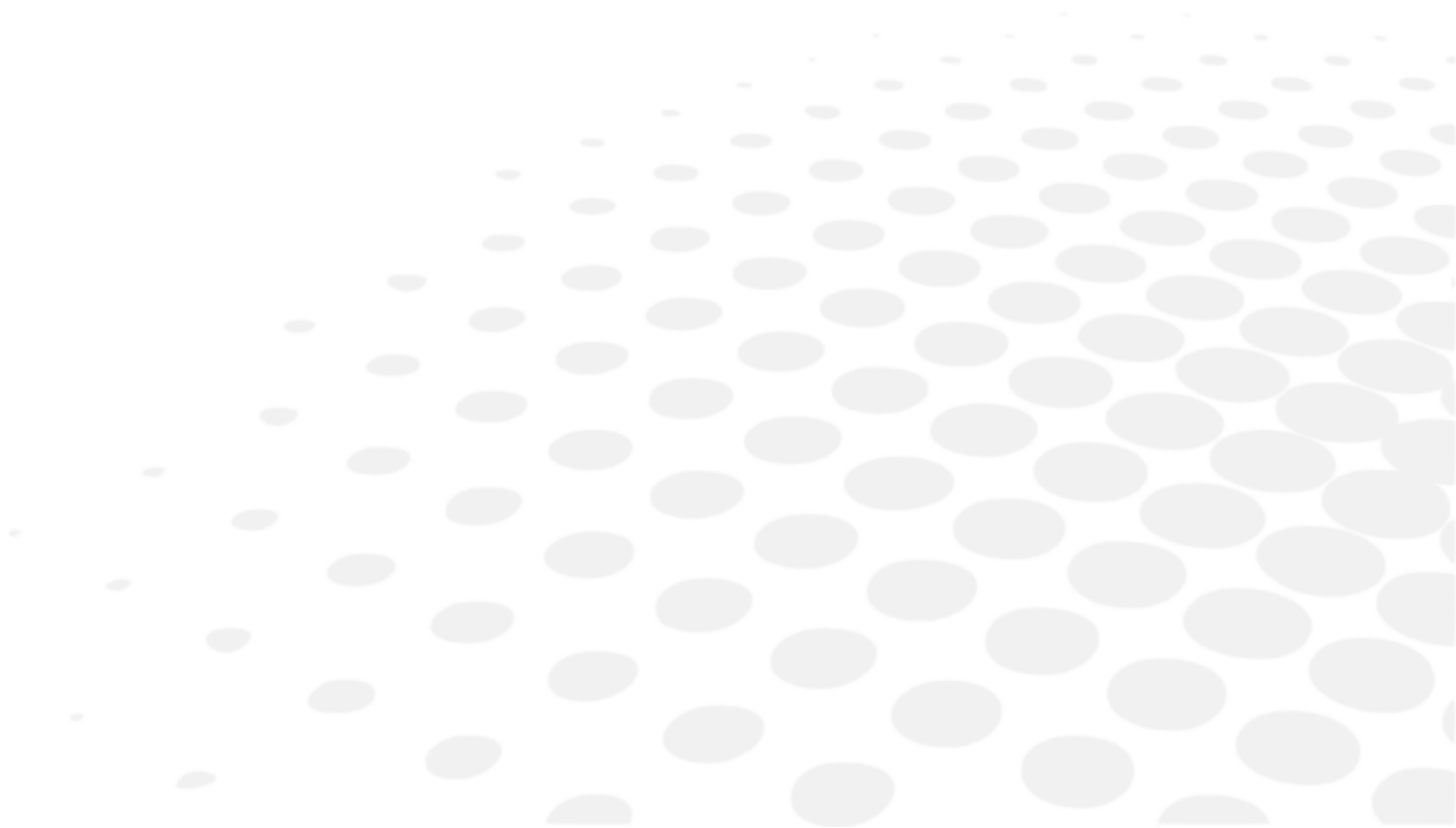
III. ACCESSORIES

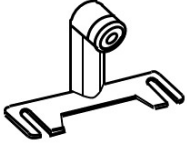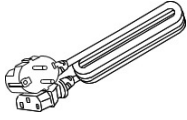

	Model Eye: 1 With a contact lens holder Diopter value is indicated on the sticker.
	Power cord: 1 (2.5m)
	Printer paper: 3 (Width: 58mm) [2 included and 1 installed in the unit]
	Fuse: 2 (T2A L 250V)
	Chinrest liner: 1 (1,000 sheets)
	Chinrest liner pin: 2
	Dust cover: 1
	Operation manual: 1


Name	Model No.	Length
Power cord	KP4819YKS31A or equivalent	2.5m

- 

Use only the accessories specified by us.
Use of the accessory (power cord) other than specified above may adversely affect other instruments and/or cause malfunction of this device.
- 

Extra care should be taken for storage of a model eye.
Do not store it at where it is dusty or high-temperature and humidity.
Avoid direct sunlight, high-temperature and humidity when storing the printer paper because it is a thermal paper.

IV. DEVICE

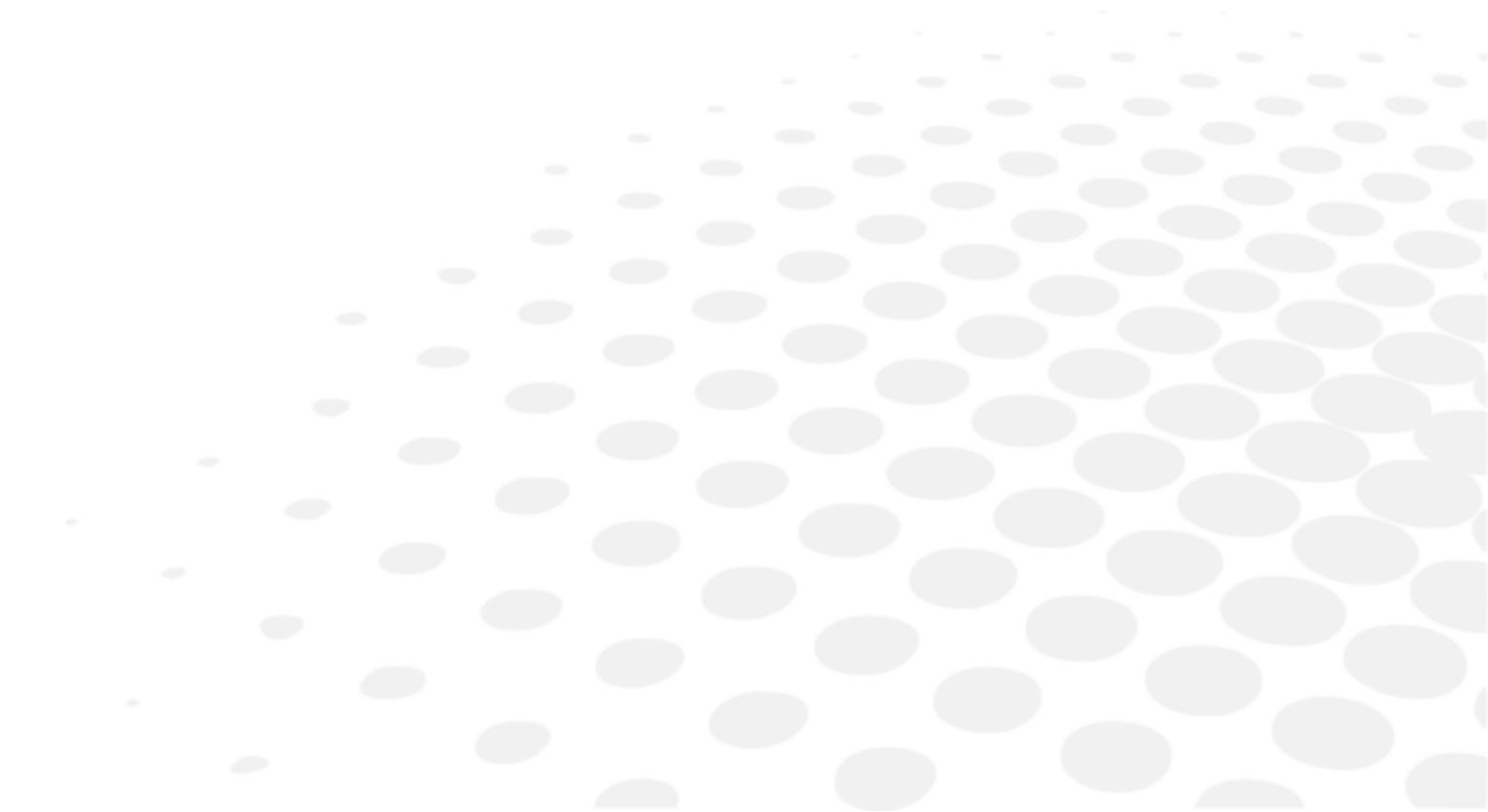

1. General description of product

This product (AKR550) aims to objectively measure the refractive power of the eye by using the light that is projected to and reflected from the eyeground. It also aims to measure the radius of corneal curvature by using the light that is projected to and reflected from the cornea.

As the feature of this device, the LCD is tilted to the vertical direction and horizontal direction so that the angle can be adjusted.

As for safety consideration, see “V. Instructions for use” of this manual.

2. Intended use defined

This product (AKR550) aims to objectively measure the refractive power of the eye by using the light that is projected to and reflected from the eyeground. It also aims to measure the radius of corneal curvature by using the light that is projected to and reflected from the cornea.

Moreover, it can measure the pupil diameter by taking the image of the anterior eye of the examinee.


3. Classification defined, rule given

This product is an active device which does not belong to the category of the non-invasive device and does not intend the following performances: supply of energy / observation of physiological process/ irradiation of ionization radiation/ medication of medicines etc.

Therefore, this is a class I medical device with a measuring function based on the rule 12 of MDD Annex IX.


4. Classification of device

According to European directive on medical device, the AKR550 is a class I with measuring function medical device.

It is marked  0459. Date of first marking is February 2016. The expected lifetime is 7 years.

Type of protection against electrical shock: Class I Equipment

Class 1 equipment is equipment in which protection against electric shock does not rely on basic insulation only, but which includes an additional safety precaution in that means are provided for the connection of the equipment to a protective earth conductor in the fixed wiring of the installation in which a way which accessible metal parts cannot become live in the event of a failure of the basic insulation.

	Degree of protection against electrical shock: Type B Equipment Type B equipment provides an adequate degree of protection against electrical shock, particularly regarding allowable leakage currents and reliability of the protective earth connection.
---	---

Degree of protection against harmful intrusion of water (IEC 60529): IPX0

This product does not provide protection against intrusion of water.

Classification by safety of use in air/ flammable anesthetic gas, oxygen or nitrous oxide/ flammable anesthetic gas atmosphere:

- Equipment not suited for use in air/flammable anesthetic gas, oxygen or nitrous oxide/flammable anesthetic gas atmosphere.
- This product should be used in an environment free of flammable anesthetic gas and other flammable gases.

Classification by operation mode: Continuous operation with short-time loading.



5. Usage of product

This product is for medical use which must be used under instructions of a doctor.


6. Mode of operation

This product is for continuous operation. It takes approx. 2 sec. for each measurement.

7. Parts identification

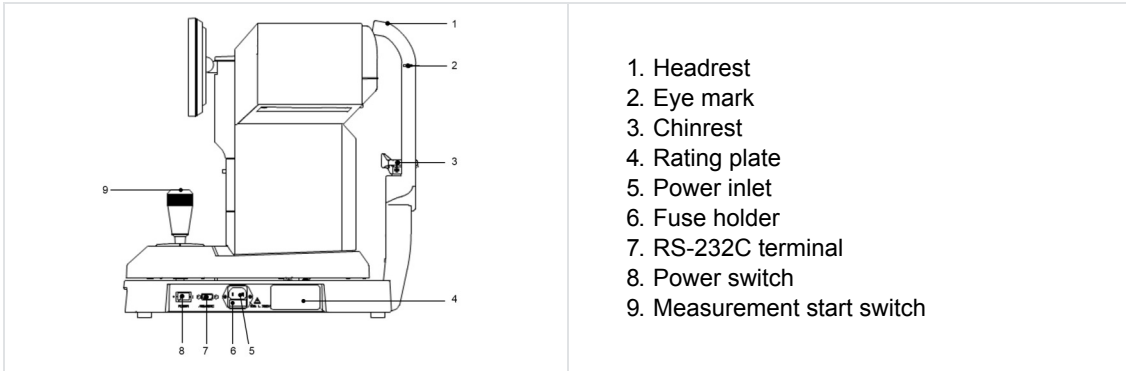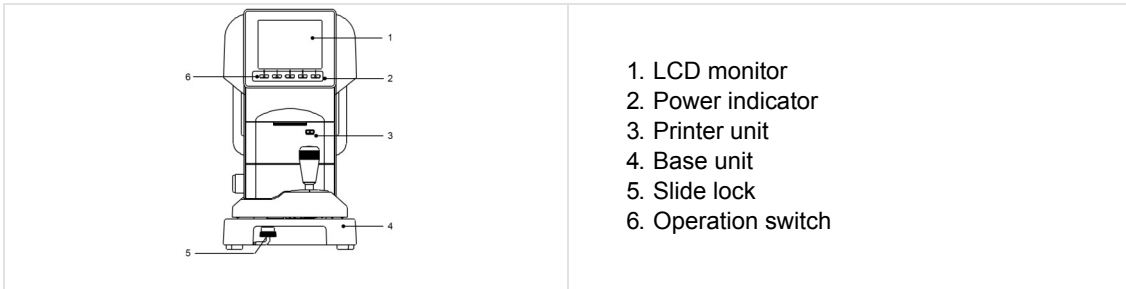

 Applied parts are headrest and chinrest.


 There is the parts list separated from this manual. Also there is another parts list related to the safety.

V. INSTRUCTIONS FOR USE



1. Conveyance

1. Before conveyance, move the main unit undermost, set it in the center of the base unit and secure it by tightening the slide lock.

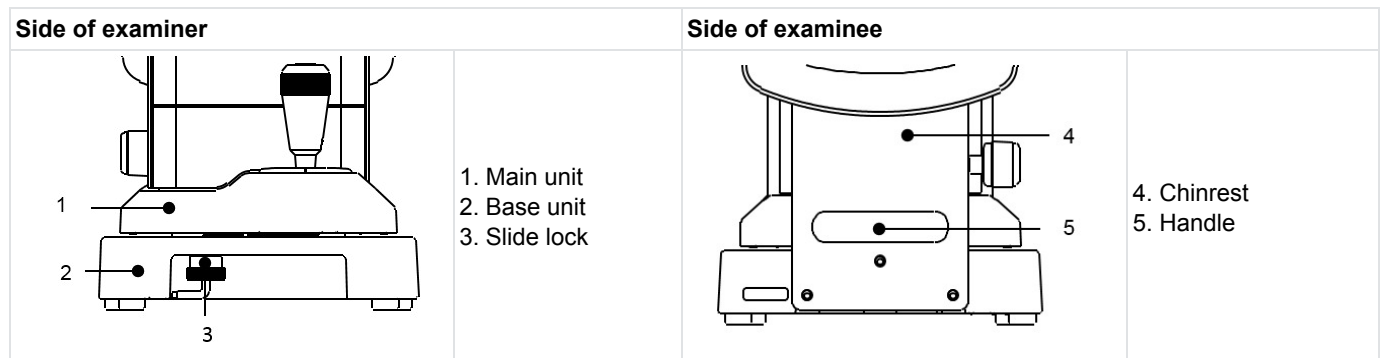


2. The slide lock can be tightened by pushing it up and rotating it in counterclockwise direction.

3. At the time of conveyance, hold the back and front of the base unit (the cutout of the front side and the handle under the chinrest) with both hands securely. Do not hold the headrest, chinrest or LCD monitor because it can result in deformation or malfunction of them.

4. Do not drag the power cord with being attached to the main unit. It can result in malfunction of the device caused by fall or drop, or personal injury if the cord is stuck or stepped on.

2. Installation

1. Do not expose the view window of the device directly to the sunlight or bright lighting from other sources.

	Great care should be taken because the measurement cannot be carried out if the examinee is exposed to strong light or glare during the measurement and his/her pupil contracts too small.
---	--

2. Do not operate it at the place where it is either dusty or grubby.


3. Environment with extremes in heat and humidity should also be avoided. In case of using the device, fulfill the environmental conditions at the time of unpacking and use.

4. Keep away from the sites which may experience strong vibrations or sudden shocks.

5. It can result in malfunction if the device is accidentally overturns. Also, it is very dangerous if it is dropped on your foot etc. Do not store it at the unstable or high place.

3. Connection / Wiring

Warning

	To avoid the risk of electric shock, this equipment must only be connected to a supply means with protective earth.
---	---

1. Connect the earth cable of the power cord to the earth terminal.

2. Do not damage the power cord (folding it small, pulling it or putting a heavy object on it etc.).

Also, do not remodel it.

Especially at the time of installation, keep enough space for the power cord to prevent it from being damaged or failure.

3. If the cord is damaged (disconnection, failure of coating etc.), replace it with the new one.

It can result in electrical shock or fire.

4. Insert the power cord in the outlet and this device securely.

If it is not connected securely, it can result in fire or electrical shock.

5. Clean the power cord all the time to avoid dust or oil etc.

It can result in malfunction or fire if the terminal unit is not clean.

6. Check if the terminal unit is dirty when the power cord becomes hot.

If it is not dirty, replace it with the new one. It can result in fire or malfunction if you keep using it.

7. Use this device with the proper power-supply voltage.

If the power-supply voltage is excess, it can result in malfunction or fire.

8. Hold the plug unit when plugging in and out.

9. Do not touch the power plug with wet hands. It can result in electrical shock.

10. Plug out the power cord when it is not used for a long time.

4. Maintenance / Inspection

1. This device is a precision optical device. Always handle it with care and do not drop it.

2. Do not touch the optical parts, such as a view window, with your hands and be sure to avoid dust because its measurement accuracy could be adversely affected.

3. Unplug the device before cleaning.



When dust or fingerprints are on the optical parts, wipe them off with a soft cloth gently. Take great care when cleaning them because they are particularly sensitive and fragile.

4. If the measurement unit cover, main unit cover or operation panel is dirty, gently wipe it with a dry cloth. For stubborn stains, a little water or neutral cleanser is recommended.



Avoid using organic solvents which will dissolve the water based paint on surface of the device.

5. Clean the chinrest and headrest with the neutral cleanser. For disinfecting the parts especially where the examinee may contact such as the chinrest and headrest, use the ethanol for disinfection.

- Ethanol for disinfection contains 76.9 to 81.4vol% of ethanol (C₂H₆O) at 15°C (specific gravity).



Do not spray chemicals on the device when disinfecting. If they get inside the device, it can cause malfunction.

Basically, it is not necessary to replace the chinrest and headrest rubber. They comply with ISO 10993-1.

6. If the device is not used for a long time, remove the power cord from the outlet.

7. When the device is not in use, protect it with a supplied dustproof cover.

If dust is adhered, it affects its measurement accuracy.

8. Never attempt to fix or remodel the device.

When the device fails to function properly, do not touch the inside.

Contact us or the place of purchase.

5. Disposal



Instructions for the disposal of the instrument in accordance with Directives 2012/19/EU and 2011/65/EU regarding the limitation of dangerous substances in electrical and electronic equipment and the disposal of electrical and electronic waste.


When it reaches the end of its lifetime, the instrument should not be thrown out with the household refuse. It can be disposed of at a waste management center operated by the municipality or the retailers who offer this service. The separate disposal of an electrical device avoids any damage to the environment or health that could result from a noncompliant disposal, and also allows the materials it is composed of to be recycled in order to save energy and resources. The pictogram of the wheeled container appears on the label of the instrument. It indicates the obligation for separate collection and disposal of end-of-life/out-of-use electrical and electronic equipment.



Battery users must not dispose of batteries as unsorted general waste, but treat properly. If a chemical symbol is printed beneath the symbol shown above, this chemical symbol means that the battery or accumulator contains a heavy metal at a certain concentration.

The lithium battery is used for the control board to store the information about the date and time. Basically it is not necessary to replace it because it is rechargeable.

VI. HOW TO USE

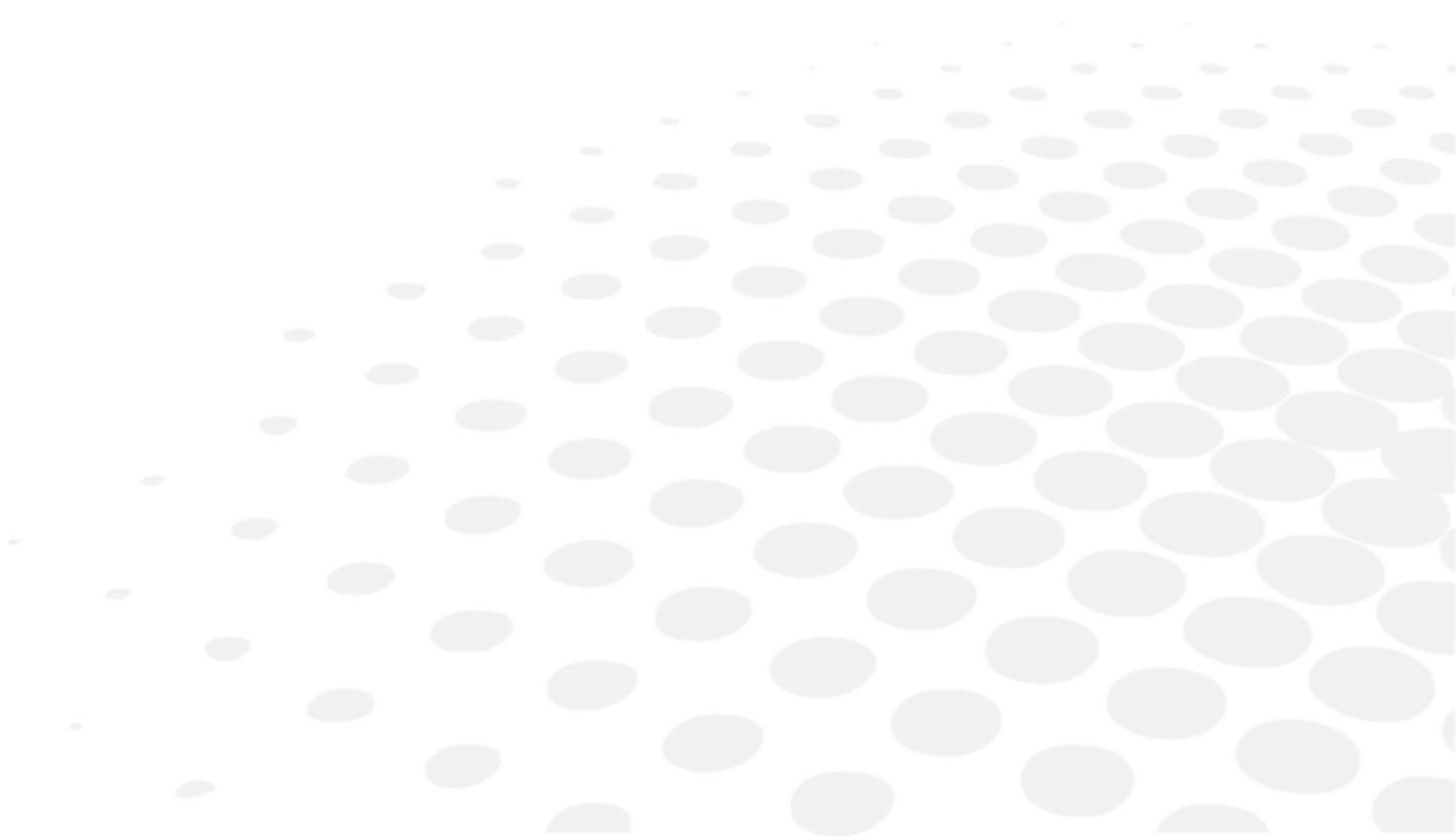

1. Operation procedure

The operation switches under the monitor correspond to the icons displayed on the bottom of the monitor.


For usual measurement, the operating switches correspond to the icons as shown below.



1: Start method switching function: The start method (START items on the [Setup] screen: [Auto-Quick/Auto/Manual] can be switched on the measurement screen by pressing and holding the setup switch.

2: Feed Function: It is switched to the feed function by pressing and holding the print switch and the paper is fed.

Operation instructions of joystick

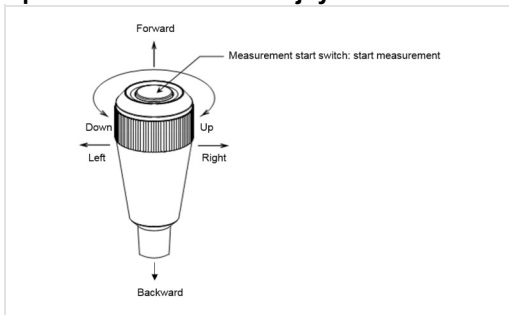

- When moving the joystick forward, the measurement unit moves to the side of the examinee.
- When moving the joystick backward, the measurement unit moves to the side of the examiner.
- When moving the joystick to the right or the left, the measurement unit moves to the right or the left (respectively).
- When rotating the joystick to the right, the measurement unit moves up and when rotating the joystick to the left, the measurement unit moves down.

2. Measurement flow

Procedure	Process	Reference section	Relevant section
1	Prepare for measurement ↓	VI > 3 > a	
2	Power distribution ↓	VI > 3 > b	
3	Ask an examinee to ready for a measurement ↓	VI > 3 > d	VI > 4 > Setting of [Setup] screen VII > 2 > Fuse replacement VII > 3 > Setting of chinrest liner
4	Alignment ↓	VI > 3 > e	VIII > Tip for effective measurement
5	Take a measurement ↓	VI > 3 > f	IX > Error display
6	Print out the measurement result ↓	VI > 3 > g	VII > 1 > Reload of printer paper
7	Switch right/left eye of the examinee OR switch the examinee ↓	Go to the procedure 3	
8	Storage of the device	VII > 4	


This device has the auto/manual measurement switch function. In case of the auto measurement, the measurement is started automatically after alignment is achieved. In case of the manual measurement, on the other hand, the measurement is started by pressing the measurement start switch.



The measurement can be started manually by pressing the measurement start switch even when the setting of Start is either [Auto] or [Auto-Quick].

3. Measurement


a. Preparation for measurement

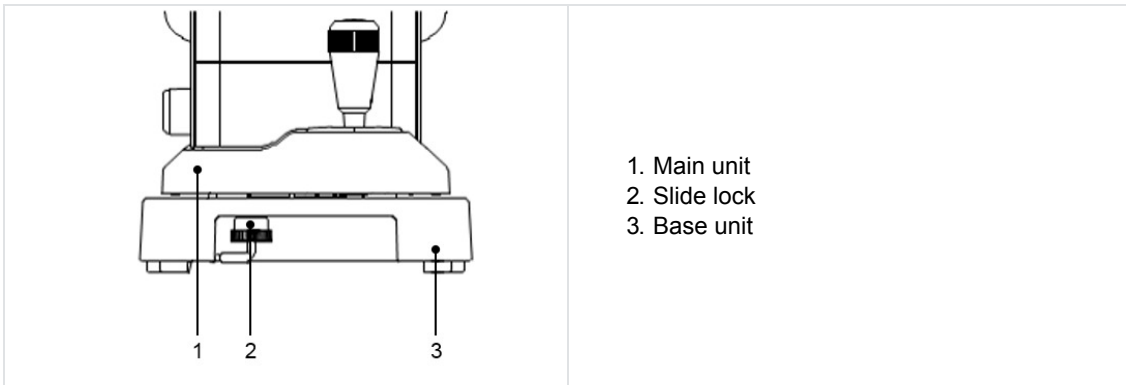

1. Do not place the device at where outside light exposes it directly from the examinee side.
2. Make sure that the printer paper, the fuse and the chinrest liner are installed properly.
3. Refer to “VII > 1 > Reload of Printer Paper”, “VII > 2 > Fuse Replacement” or “VII > 3 > Setting of Chinrest Liner” of “VII. Storage and Maintenance” of this manual for the installation procedures of the parts of (2) above.
4. After distributing the power, rotate the slide lock of the main unit (under the base unit) and release the main unit.

b. Power distribution


<p>POWER RS-232C</p>	<p>1. Make sure that the power switch of the main unit is OFF (○).</p>
	<p>2. Insert the power cord in the power plug connector of the main unit and insert the power plug in a socket.</p> <p>⚠: Always make sure that the cable is grounded.</p> <p>⊘: Do not use an additional power strip or an extension cord.</p>

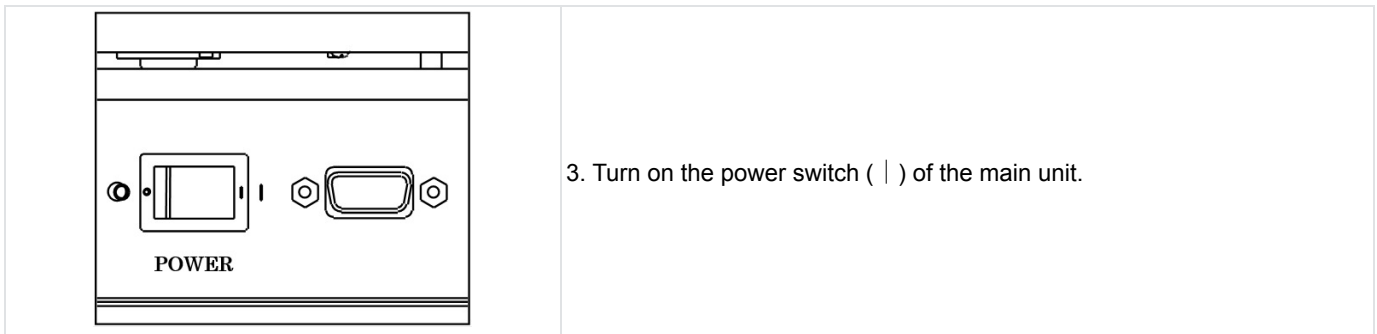

3. Turn on the power switch (|) of the main unit.

c. Standby

When the power is turned on, the screen as shown below appears on the LCD monitor, which is ready for the measurement.

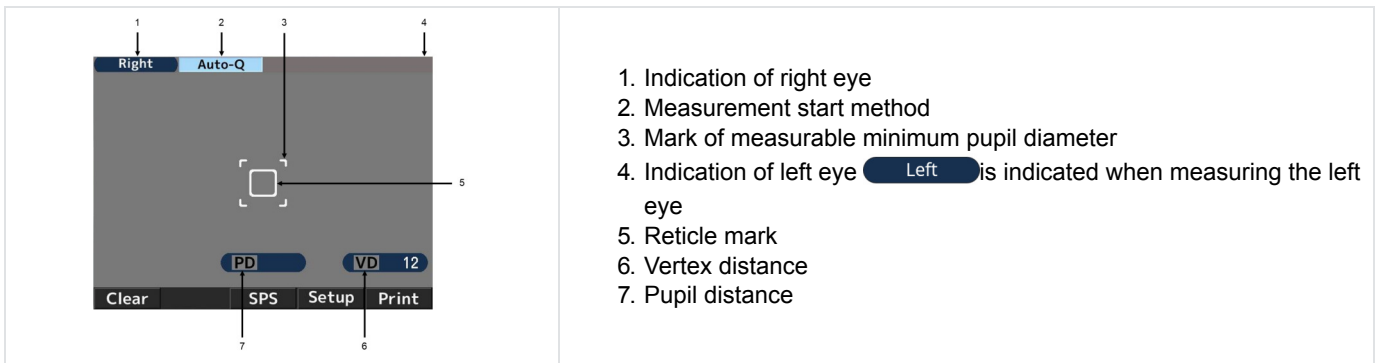

1. Indication of right eye
2. Measurement start method
3. Mark of measurable minimum pupil diameter
4. Indication of left eye **Left** is indicated when measuring the left eye
5. Reticle mark
6. Vertex distance
7. Pupil distance

Icon	Function
Right	Indicate the eye (right or left) in process of measurement.
Left	
Auto-Q Auto	Indicate the measurement start method.
VD 12	Indicate the vertex distance. It can be switched between 0, 10, 12, 13.5 and 15mm.
Clear	Clear the measurement results (values).
IOL	Switch ON and OFF of the IOL mode.
R/K	It switches the measurement mode. There are 4 measurement modes: refractive and keratometry continuous measurement, refractive measurement, keratometry measurement and scotopic pupil size measurement.
Setup	It is switched to the [Setup] screen.
Print	Display and print out the measurement result.

d. Preparation of examinee

1. Clean the chinrest and dispose one chinrest liner on the top.


Clean the chinrest with the neutral cleanser when there is no chinrest liner.
For disinfecting the chinrest, use the ethanol.

- Ethanol for disinfection contains 76.9 to 81.4vol% of ethanol (C₂H₆O) at 15°C (specific gravity).

2. Ask the examinee to put his/her chin on the chinrest. Adjust the height of the chinrest so that the examinee's eye level is in line with the eye mark.

3. Uncomfortable posture may fatigue the examinee during the measurement. Adjust the chinrest or the device to avoid it.


4. It affects its measurement accuracy if the examinee moves his/her head during the measurement. Ask him/her to fix his/her forehead on the headrest and look at the target with a good posture.
5. Talk to the examinee fluently and try not to make him/her nervous.

 Uncomfortable posture may fatigue the examinee during the measurement. Adjust the height of the optical table or the chair to avoid it.


e. Alignment


There are 3 types of startup procedures [Auto Quick, Auto and Manual] for AKR550. It can be switched in start of the [Setup] screen.

In case of [Auto Quick] or [Auto]


 It starts measurement automatically when it brings the subject eye into focus.

1. Look for the subject eye by operating the joystick.
The Kerato ring appears as bringing it into focus.

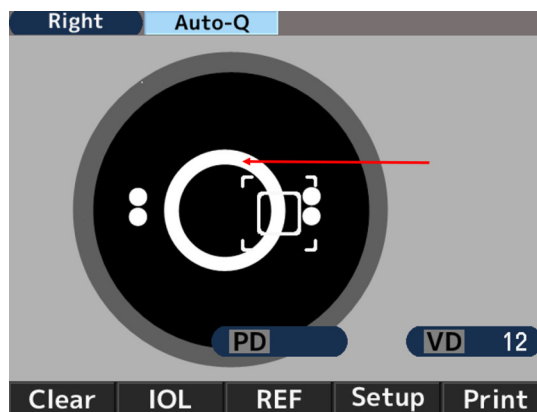


 If the eyelid is over the Kerato ring, urge the examinee to open his/her eye wider.

2. The alignment mark (+) will appear as aligning the reticle mark with the center of the pupil of the subject eye and bringing it into focus. Operate the joystick in such a way that the alignment mark (+) comes to the center of the reticle.

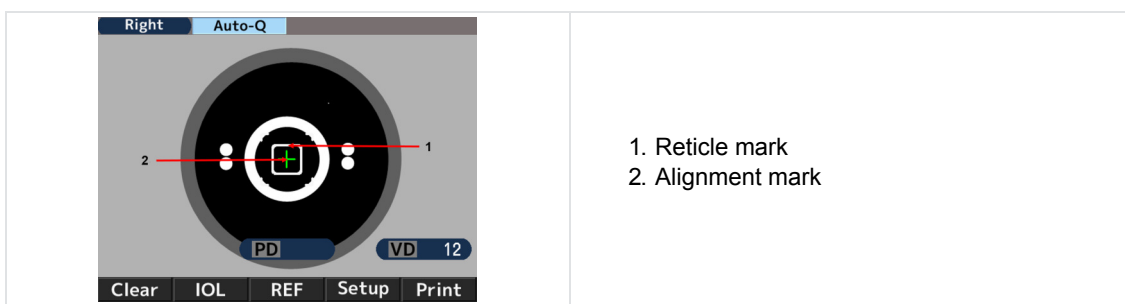

3. Operate the joystick so as to bring it into focus as aligning the alignment mark (+) with the center of the reticle mark. The measurement is started when alignment is achieved and the mark of the measurable minimum pupil diameter is changed to green.

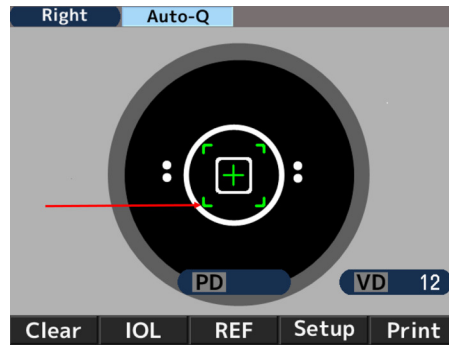


4. The measurement values are displayed when the measurement is finished. The arrows are displayed when the specific times of measurement is finished. Move the main unit to the direction of the arrows and take a measurement of the other eye.




In case of [Manual]

1. Look for the subject eye by operating the joystick.
The Kerato ring appears as bringing it into focus.

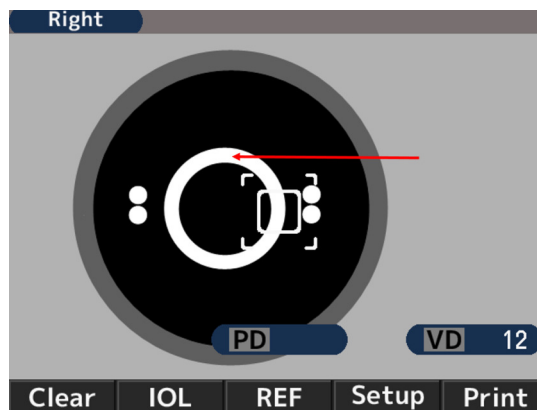


 If the eyelid is over the Kerato ring, urge the examinee to open his/her eye wider.

2. The alignment mark (+) will appear as aligning the reticle mark with the center of the pupil of the subject eye and bringing it into focus. Operate the joystick in such a way that the alignment mark (+) comes to the center of the reticle.

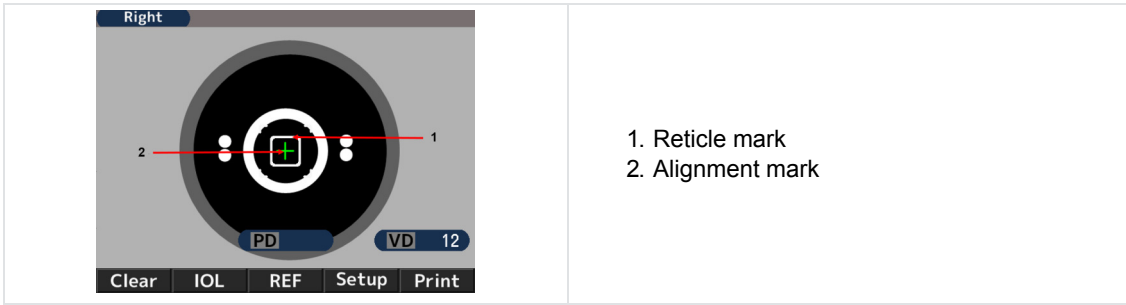

1. Reticle mark
2. Alignment mark

3. Operate the joystick so as to bring it into focus as aligning the alignment mark (+) with the center of the reticle mark. Start the measurement when alignment is achieved and the mark of the measurable minimum pupil diameter is changed to green.

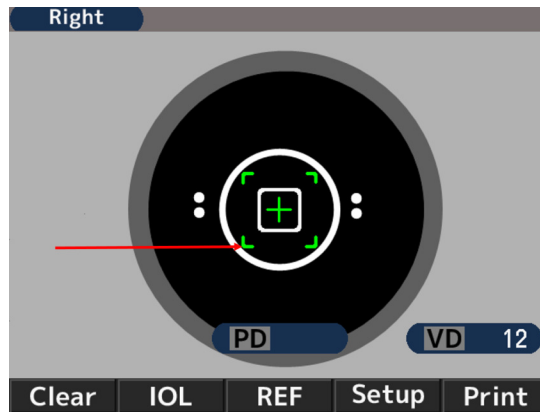

f. Measurement

The measurement start method is different depending on the setting.

Setting	Measurement start method
The setting of start is either [Auto-Quick] or [Auto]	The measurement is started automatically when alignment is achieved.
The setting of start is [Manual]	Start the measurement by pressing the start switch when alignment is achieved.



1. Number of refractive measurement

2. Refractive measurement value

- S: Spherical value
- C: Cylindrical value
- A: Axis angle

3. Number of Kerato measurement

4. Kerato measurement value

- R1: Radius of curvature (Max.)
- R2: Radius of curvature (Min.)
- AX: Axis Angle

5. Photopic pupil diameter measurement result

6. Vertex distance

7. Pupillary distance

Far vision

8. Pupillary distance

Near vision



The PD value is indicated after the refractive power of both right and left eyes are measured.

The order of the eye to be measured is not important.

The NPD value is indicated only if the number of [W-D (cm)] on the [Setup] screen is set.

g. Print-out of measurement result

The measurement result can be printed out by pressing the print switch after the measurements.

A maximum of the data for each eye can be saved and the most reliable value among them is indicated as the optimum value. The optimum value is printed out only when more than three times of measurement is taken for each eye. The format of the output [All, All/Eco, Eco or OFF] can be set on [Print REF/KRT] on the [Setup] screen.

- **[All]**: Print out a maximum of ten data of the refractive measurement and Kerato measurement for each eye.
- **[All/Eco]**:
 - Print out a maximum of ten data of the refractive measurement for each eye.
 - Print out only the optimum values for the Kerato measurement.
- **[Eco]**: Print out only the optimum values for all of the measurement.
- **[Off]**: Print out no data.

<Sample of Printout 1>

Print [REF/KRT] setting : Eco

NAME				1
2011 11 22				14:30
2 VD=12				
<R>	SPH	CYL	AX	
	- 3.87	-0.75	172	
3				
<R>	mm	D	AX	
R1	8.33	40.50	175	
R2	8.20	41.12	85	
AVE	8.26	40.75		
CYL		-0.62	175	
<L> SPH CYL AX				
- 3.75 -1.12 14				
4				
<L>	mm	D	AX	
R1	8.37	40.37	8	
R2	8.12	41.50	98	
AVE	8.25	40.87		
CYL		-1.13	8	
PD = 70				
AKR550				

1. Date and time of the measurement

2. Refractive measurement result (optimum value)

- o SPH: Spherical value
- o CYL: Cylindrical value
- o AX: Axis angle

3. Kerato measurement result (optimum value)

- o R1 : Radius of curvature (Max.)
- o R2: Radius of curvature (Min.)
- o AVE: Average of R1 and R2
- o CYL: Cylindrical value

4. Pupillary distance

<Sample of Printout 2>

Print [REF/KRT] setting: All

1. Message area
 2. No. 00001
 3. NAME 2011 11 22 14:30
 4. D=12
 5. R> SPH CYL AX PPS
 - 3.75 -0.75 172 6.6
 - 3.87 -0.75 170 6.5
 - 3.87 -0.62 174 6.6
 6. SE - 3.87 -0.75 72 6.6
 7. SE - 3.98 SPS 8 7.9
 9. R> mm D AX
 R1 8.43 40.00 9
 R2 8.21 41.12 99
 AVE 8.32 40.62
 CYL -1.12 9
 R1 8.43 40.00 10
 R2 8.22 41.12 100
 AVE 8.32 40.50
 CYL -1.12 100
 R1 8.30 40.62 2
 R2 8.16 41.37 92
 AVE 8.23 41.00
 CYL -0.75 2
 10. R1 8.31 40.62 180
 R2 8.17 41.37 90
 AVE 8.24 41.00
 CYL -0.75 180
 11. REST -0.12 90
 <L> SPH CYL AX PPS
 - 3.75 -1.12 13 6.6
 - 3.75 -1.12 15 6.6
 - 3.75 -1.12 14 6.6
 - 3.75 -1.12 14 6.6
 SE - 3.99 SPS 7.9
 12. PD = 65 13. NPD = 62 (50)
 AKR550

1. Message area
2. No. of examinee
3. Data of right eye
4. Refractive data
5. Photopic pupil size
6. Optimum values of the refractive measurement results
 They are indicated when more than 3 times of measurements are taken for each eye
7. Spherical equivalent
8. Scotopic pupil size
9. Kerato data
10. Optimum values of the corneal curvature radius
 They are indicated when more than 3 times of measurements are taken for each eye
11. Residual astigmatism
12. PD for far vision
13. PD for near vision

Message area


It can print out the registered characters in the range of 24 characters/line × 2 lines in the message area. Refer to the [Message] section of “VI > 4 > Setting of [Setup] screen” about registration of the characters.

4. Setting of [Setup] screen

The standard measurement mode is preset to be ready for use.

However, altering the setting can be done easily if necessary.

Press **Setup** switch under the LCD monitor and display the [Setup] screen.

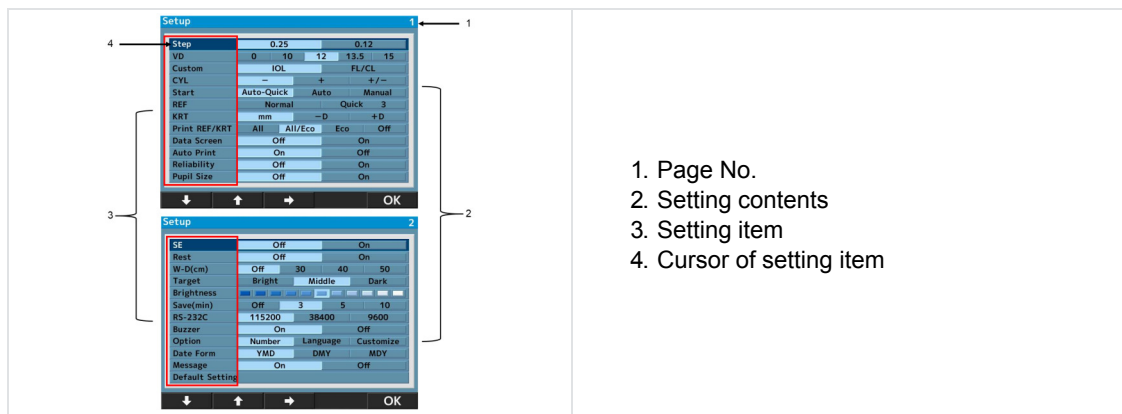

1. Page No.
2. Setting contents
3. Setting item
4. Cursor of setting item

There are 24 setting items on the menu screen.

Select the item to be changed by pressing **↓** or **↑** and change it by pressing **→**.

After changing it, return to the measurement screen by pressing **OK**.

Details of each setting item - [Screen 1]

- **[Step]:** Select the step for refractive measurement.
- **[VD]:** Select the corneal vertex distance.
- **[IOL]:** Select the function of the operation switch.
 - [IOL]: Switch to the mode to measure.
 - [IOL. FL/CL]: Switch the corneal vertex distance (frame value/ contact value).
- **[CYL]:** Select the sign of cylindrical value.
- **[Start]:**

Select the measurement start method.

 - [Auto-Quick]: It starts measurement when the alignment is achieved. Take 1 time of Kerato measurement and 3 times of refractive measurements continuously for each eye.
The result is printed out automatically when [Auto Print] is set as [ON]. (For the refractive measurement, only one time of the fog control is done at the beginning).
 - [Auto]: Take 3 times of Kerato measurements and refractive measurements continuously for each eye.
The result is printed out automatically when [Auto Print] is set as [ON]. (For the refractive measurement, the fog control is done every time).
 - [Manual]: Measurements are taken every time the measurement switch is pressed.
- **[REF]:** Select the refractive measurement method. The setting is valid only when the measurement start method is set as manual.
 - [Norma]: A measurement is taken one time by pressing the measurement start switch.
 - [Quick]: Continuous measurement is started as much as it is set by pressing the measurement start switch one time. (Maximum of 10 times.) (For the refractive measurement, only one time of the fog control is done at the beginning).
- **[KRT]:** Select the sign of Kerato measurement result.
 - [mm]: Corneal Curvature radius
 - [- D]: corneal astigmatism (-)
 - [+D]: corneal astigmatism (+)

- **[Print REF/KRT]:** Select the format of print-out.
 - [All]: Print out all of the measurement data.
(Maximum of 10 times for each eye).
 - [All/Eco]: Print out all of the REF measurement.
(Maximum of 10 times for each eye).
Print out only the optimum values for the Kerato measurement.
 - [Eco]: Print out only the optimum values.
 - [Off]: No measurement result is printed out.
- **[Data Screen]:** Display the stored measurement results.
 - [On]: Display the measurement results on the screen.
 - [Off]: Display no measurement result on the screen.
- **[Auto Print]:** Select the print-out method.
This function is valid only when the setting of Start is either Auto-Quick or Auto.
 - [On]: Activate the auto print function.
 - [Off]: Invalidate the auto print function.
- **[Reliability]:** Select if displaying the low reliability mark on the measurement values or not.
 - [On]: If it is judged that the measurement value possesses low reliability, display the low reliability mark [*] on it.
 - [Off]: No low reliability mark is displayed.
- **[Pupil Size]:** Set the function of the photopic pupil diameter measurement.
 - [On]: Take a measurement of the photopic pupil diameter when taking a refractive measurement.
 - [Off]: Photopic pupil diameter is not measured.


Details of each setting item - [Screen 2]

- **[SE]:** Set the output of SE value.
 - [On]: Output the representative value of SE on print-out, data screen and communication output (XML format only).
 - [Off]: No output of SE value.
- **[Rest]:** Select the output of residual astigmatism.
 - [On]: Display the residual astigmatism.
 - [Off]: No astigmatism is displayed.
- **[W-D (cm)]:** Set the work distance.
The near pupil distance is automatically computed after the measurement and displayed on the screen.
- **[Target]:** Select the brightness of the target.
 - [Bright]: Brighten the target.
 - [Middle]: Normal setting.
 - [Dark]: Darken the target.
- **[Brightness]:** Adjust/ change the brightness of the LCD monitor.
- **[Save (min)]:** Select the switchover time to activate the power saving function (unit is min.).
- **[RS-232C]:** Select the baud rate when sending the measurement data to the exterior computer.
- **[Buzzer]:** Set if activating the buzzer at the time of switching to the power saving function or not.
 - [On]: Buzzer is ON.
 - [Off]: Buzzer is OFF.
- **[Option]:** It is switched to each option screen when selecting the item to be set on the option of the [Setup] screen.


The screen of each option and the details.



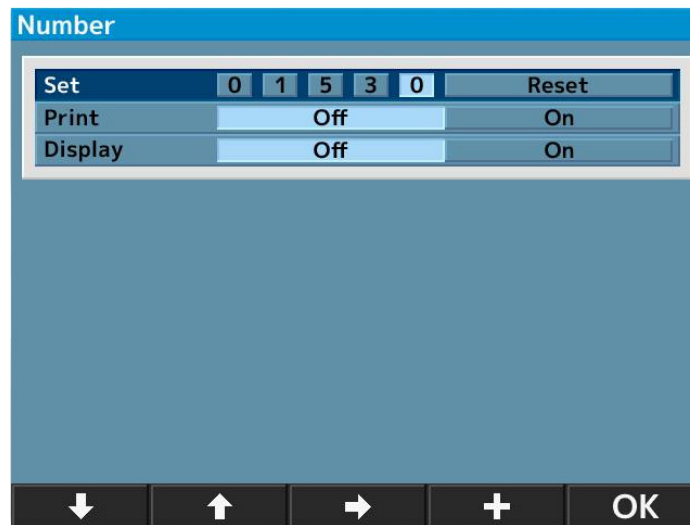


a. [Number]

This function can set or change the number of the examinee, and select if displaying the number on the monitor and the printout.



- **[Set]:** Set/change the number of the examinee.
(Maximum of 5 digits can be input).
- **[Print]:** Select if printing out the number of the examinee or not.
 - [Off]: The number is not printed out.
 - [On]: The number is printed out.
- **[Display]:** Select if displaying the number of the examinee on the screen or not.
 - [Off]: The number is not displayed.
 - [On]: The number is displayed.


 <small>NOTE</small>	<p>Reset of the number of the examinee If moving the cursor to [Reset] on [Set], the [+] switch on the bottom is changed to [Reset]. So, press the [Reset] switch to reset the number.</p>
--	---




1. Move the cursor to the item to be set or changed by pressing  or  and change it by pressing  or .
2. Go back to the [Setup] screen by pressing  after the setting or the change.

b. [Language]


This function can select the language displayed on the screen.

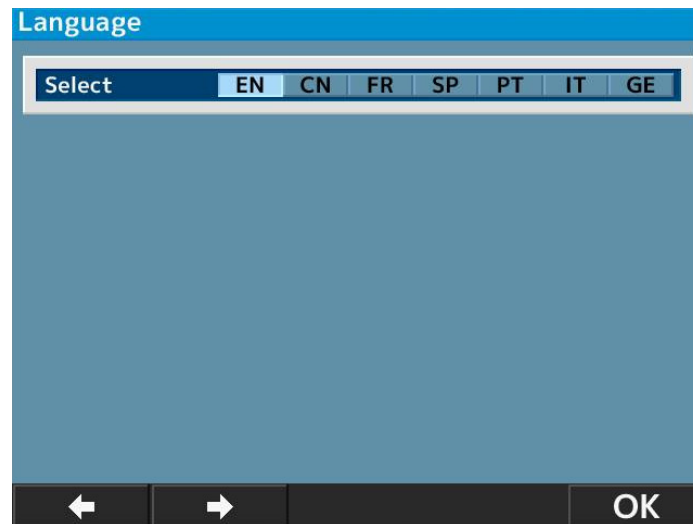Selectable language: EN (English), CN (Chinese), FR (French), ES (Spanish), PT (Portuguese), IT (Italian), GE (German).



1. Move the cursor to the item to be set by pressing  and execute it by pressing .
2. Go back to the [Setup] screen by pressing  after finishing the setting.

c. [Customize]

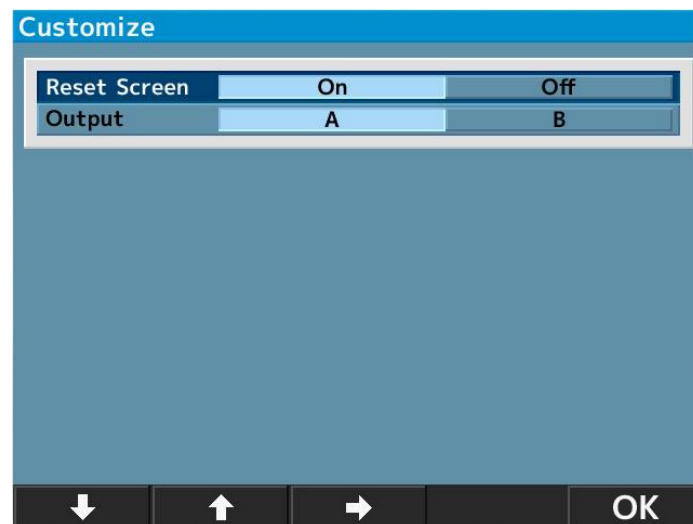


- **[Reset Screen]:** This function can delete the measurement values on the screen after printout.
 - [On]: Delete the measurement values on the screen after printout.
 - [Off]: Leave the measurement values on the screen after printout.
- **[Output]:** This function can select the output procedure of the measurement data.
 - [A]: Standard.
 - [B]: Output data common spec. of ophthalmic testing device.

(Established by Japan Ophthalmic Instruments Association)

d. [Date form]

Select the display format of the date from the followings:

- [YMD]: Display the date as year/ month/day.
- [DMY]: Display the date as day/month/year.
- [MDY]: Display the date as month/day/year.

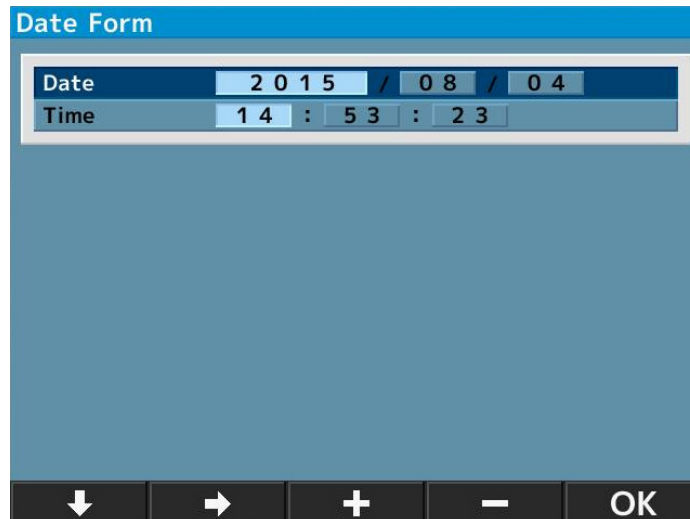


The screen above appears when selecting [YMD] and pressing **Enter**.






1. Move the cursor to the item to be changed by pressing **↓** or **→** and input the date by pressing **+** or **-**.
2. Go back to the [Setup] screen by pressing **OK** after finishing the setting.

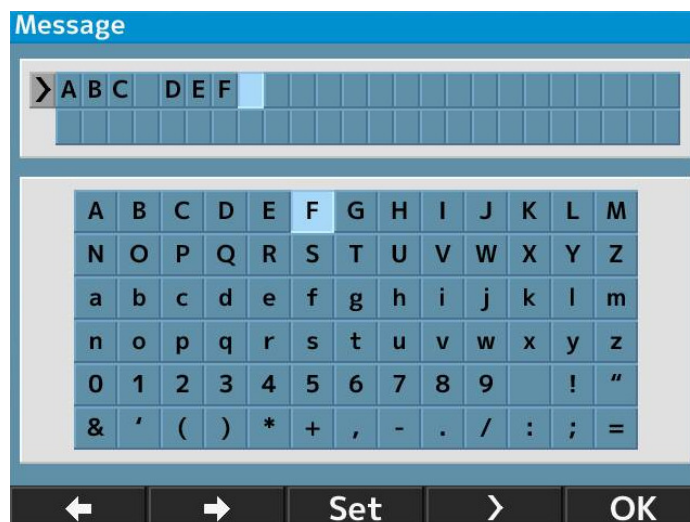e. [Message]

This function is to input the message in the range of 24 characters/line × 2 lines and output it.



The message input screen appears by selecting [On] and pressing **Enter**.

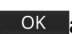1. Select the characters by pressing  or  and input them by pressing .
- A space can be input by pressing .
2. Go back to the [Setup] screen by pressing  after finishing the setting.

f. [Default setting]

Reset the settings to the factory setting.


5. Scotopic Pupil Size (SPS) - measurement function

This function is to measure the pupil size of the subject eye in the dark.

Switch to the SPS measurement by pressing the measurement mode switch on the front panel.

When measuring the scotopic pupil size, darken the room.

Indication of SPS measurement mode

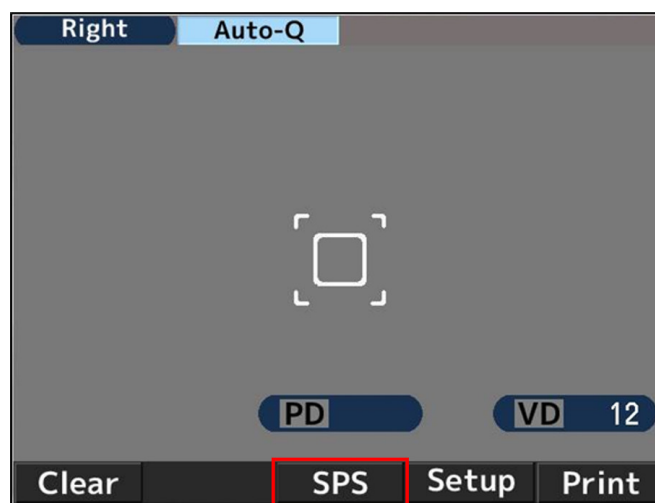

<To print out the measurement results of SPS, R/K, REF and KRT at the same time>
The measurement results of SPS, R/K, REF and KRT can be printed out at the same time by pressing the print button after the measurement of SPS when switching to SPS measurement mode without printing out their measurement result with the setting of [Auto Print OFF].

Example of printout	Example of data screen output
<pre> NAME 2011 11 22 14:30 VD=12 <R> SPS 7.3 <L> SPS 7.5 PD = 63 AKR550 </pre>	


6. IOL measurement function

When measuring the IOL (intraocular lens) implanted eye, the eye with a cataract, or the eye with the scratches on the cornea, the measurement errors may occur and it is difficult to complete the measurement with REF measurement.

In this case, it is easier to measure if moving the device closer to the examinee. Also these can be measured with IOL mode.

1. Activate the IOL function by pressing the IOL switch on the front panel of the main unit and switch to the IOL measurement mode.

At this time, the icon of the IOL measurement mode is indicated on the top of the monitor.

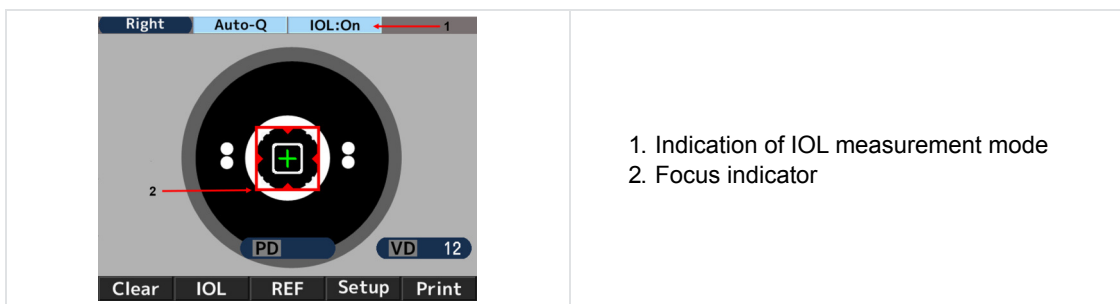



2. Catch the subject eye on the monitor by operating the joystick. As bringing the subject eye into focus, the Kerato ring, the alignment mark [+], and the focus indicator appear.

3. Operate the joystick by following the guidance of the focus indicator and move the main unit to bring the subject eye into focus.

4. It is in focus when the focus indicator turns to green. When it turns to green, take a measurement by pressing the measurement switch.



Measurement is started automatically when the setting of [Start] is either [Auto-Quick] or [Auto].

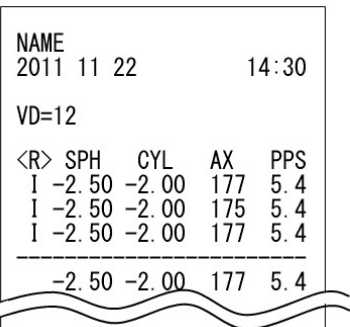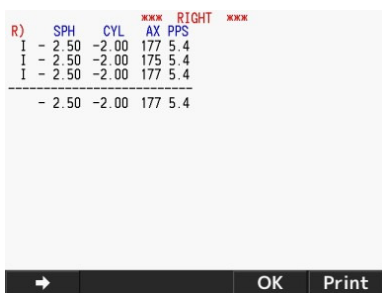Example of printout	Example of data screen output
 <pre> NAME 2011 11 22 14:30 VD=12 <R> SPH CYL AX PPS I -2.50 -2.00 177 5.4 I -2.50 -2.00 175 5.4 I -2.50 -2.00 177 5.4 ----- -2.50 -2.00 177 5.4 </pre>	 <pre> R) SPH CYL AX PPS RIGHT I -2.50 -2.00 177 5.4 I -2.50 -2.00 175 5.4 I -2.50 -2.00 177 5.4 ----- -2.50 -2.00 177 5.4 </pre>



[I] is indicated on the left of the measurement value when measuring it in the IOL measurement mode.

The IOL measurement mode is canceled by performing one of the followings:

1. Pressing the IOL switch one more time
2. Switching the measurement mode
3. Pressing the print switch
4. Turning off the power

When the measurement cannot be completed because of the errors with IOL mode.

There is a possibility that the measurement of the IOL (intraocular lens) implanted eye cannot be completed because of the implanted IOL.



In this case, move the device closer to the examinee with keeping the alignment in-focus. It might help curbing the influence and the measurement can be done.



The image of the eyeground is displayed by holding IOL or FL/CL switch for seconds.


7. Low reliability mark display function

This device has the low reliability mark display function. The low reliability mark is displayed on the measurement result which reliability is low when taking the refractive measurement with this function activated. Consider the refractive measurement value with the low reliability mark as reference.

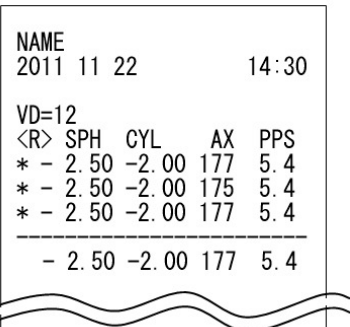
Example of printout	Example of data screen output
 <pre> NAME 2011 11 22 14:30 VD=12 <R> SPH CYL AX PPS * -2.50 -2.00 177 5.4 * -2.50 -2.00 175 5.4 * -2.50 -2.00 177 5.4 ----- -2.50 -2.00 177 5.4 </pre>	 <pre> R) SPH CYL AX PPS RIGHT * -2.50 -2.00 177 5.4 * -2.50 -2.00 175 5.4 * -2.50 -2.00 177 5.4 ----- -2.50 -2.00 177 5.4 </pre>


8. Output


This device is connected to PC etc. by RS232C.

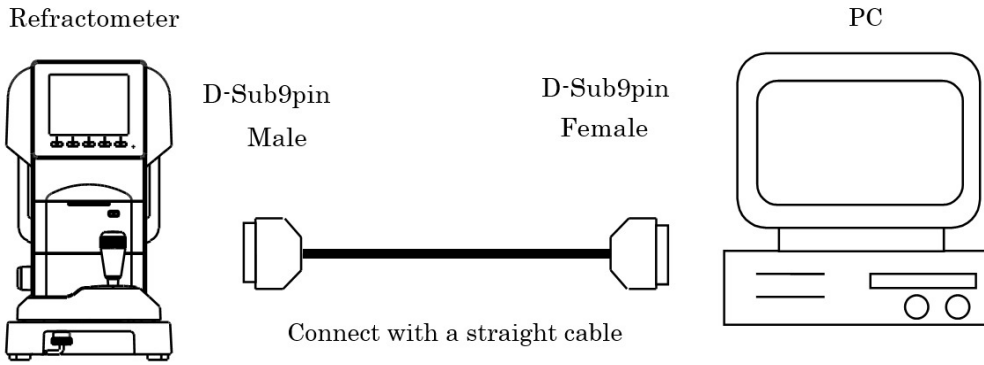

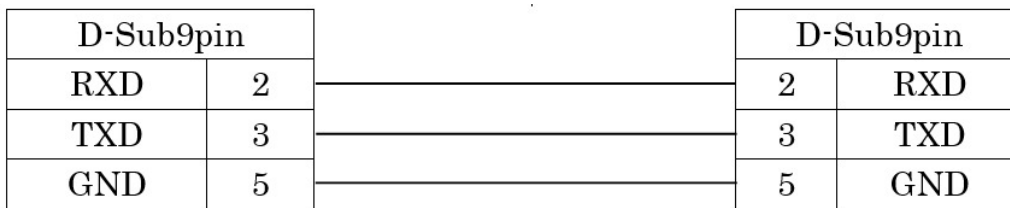

Connecting diagram: RS232C



 Use the shielded wire for the connection cable to protect the output data from noise.


 Contact your local distributor about operation, connection method and output data etc.

 The instruments which are connected to this device by RS232C should comply with the safety standard of IEC60601-1.

 Do not touch the external connection terminal and examinee at the same time. It may cause electric shock.

Select the baud rate of RS232C from below.

Selectable baud rate	Setting before shipment
115200 bps	○
38400 bps	
9600 bps	

 In case of RS232C, [Character] (data bit count), [Parity] (check of transfer data) and [Stop bit] (exit code) are set as [Character] (8), [Parity] (none) and [Stop bit] (1) and cannot be changed.

9. Data screen function

The measurement results can be displayed on the screen and checked by using the data screen function.

In case of displaying measurement results


1. Set [Data Screen] on the [Setup] screen as [On].

10. Power saving function

The power saving function is activated when leaving it on without any switch operations.

(Refer to [Save (min.)] of "VI > 4 > Setting of [Setup] Screen" about the selection of the power saving function.)

The measurement mode is activated by pressing the switch (the switch on the front panel of the measurement start switch).

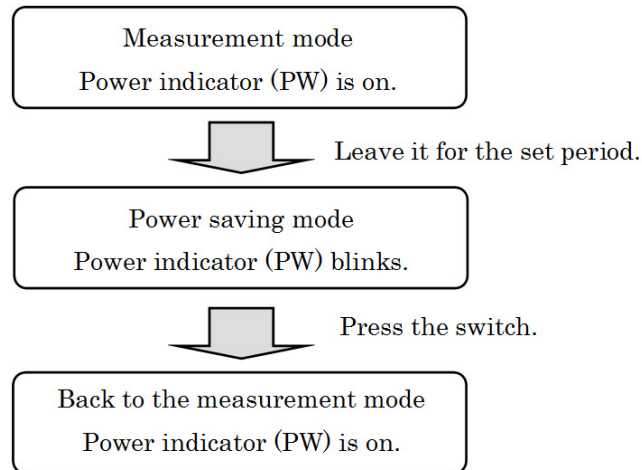

11. Contact lens: measurement of base curve

This device can measure the base curve of the hard contact lens.


The lens can be measured by placing it onto the contact lens holder of the model eye shown as below.

1. Put a small amount of water on the concave side of the contact lens holder.
2. Place the contact lens so that its convex side faces the holder.



3. Confirm that the contact lens is firmly adhered to the holder with water and does not slip down. Then, take a measurement by setting the model eye unit on the main unit.

VII. STORAGE AND MAINTENANCE

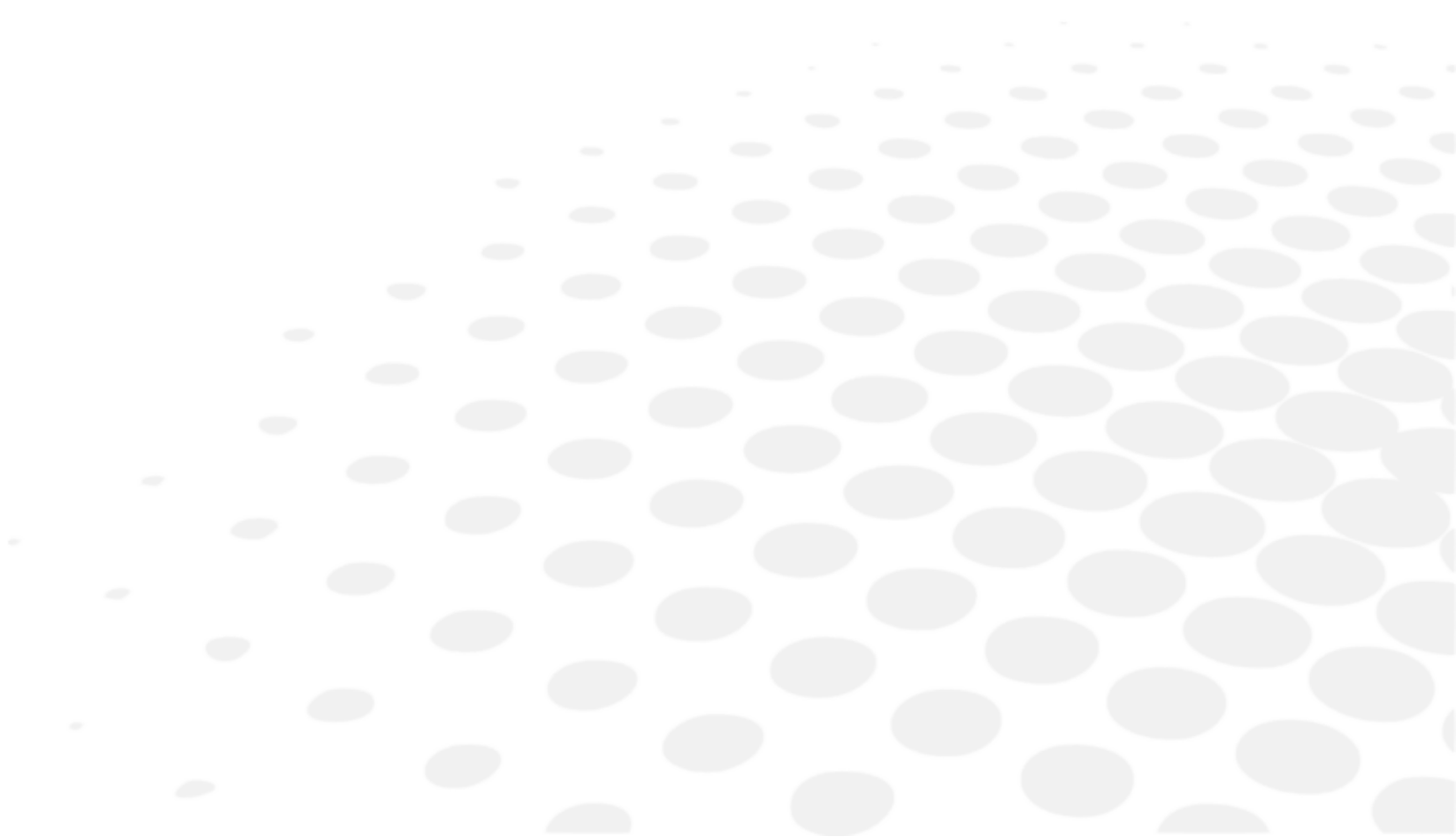





Do not perform any maintenance when used with a patient.

1. Reload of printer paper

- 1 Push on the printer door button to open the printer paper cover.

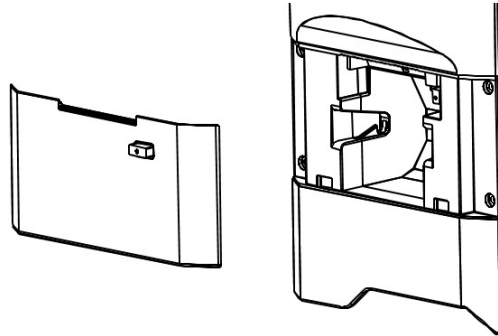


- 2 Pay attention to the direction of the paper rolled and set it in.

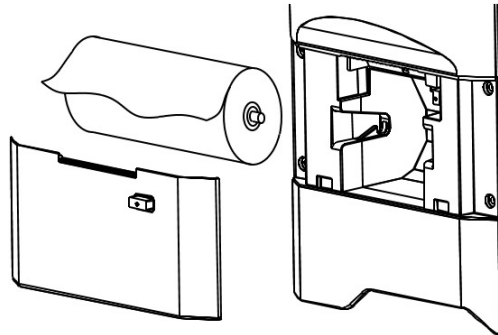

Set the paper as coming out toward the front from the upside.

- 3 Close the printer cover until it clicks.
If the cover is not closed completely, the error message appears and cannot print out.

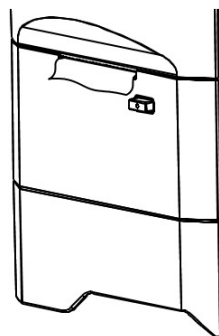


2. Fuse replacement



Unplug the power cord from the unit before removing the fuse holder. You may be in danger of electric shock if removing the fuse holder without unplugging the power cord.


When a fuse is blown, remove the fuse holder from the device for replacement.
Remove it by pushing the fuse holder and rotating it in counterclockwise direction.

! Always use the specified fuse (T2A L 250V).

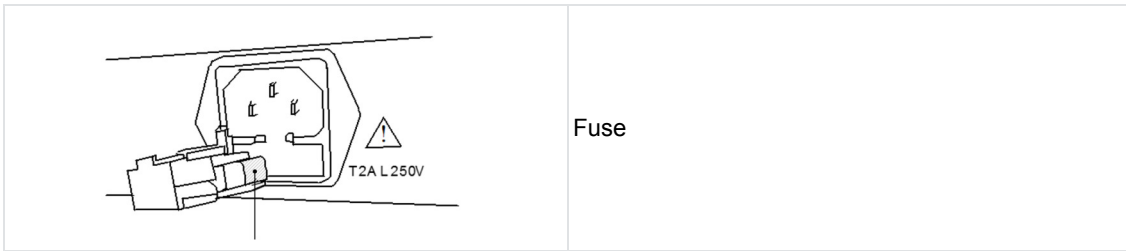

3. Setting of chinrest liner

Set the chinrest liners onto the chinrest, and secure them with the chinrest liner pins.

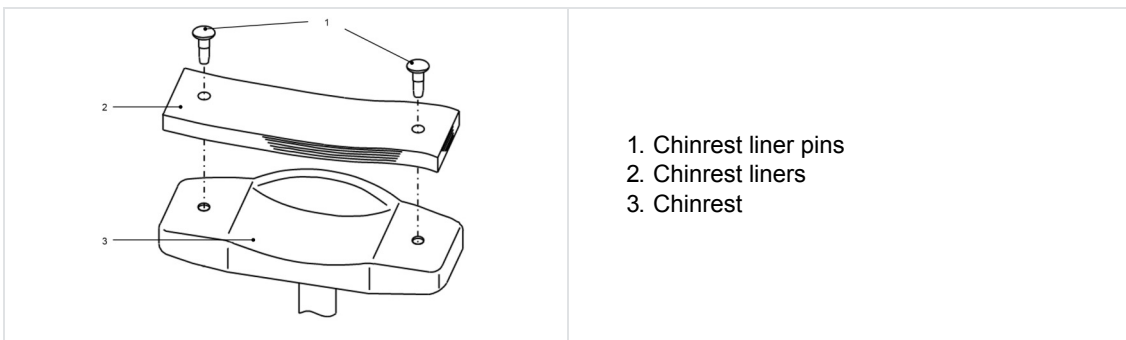

NOTE For sanitary reasons, dispose the top chinrest liner after every patient.

!

- Comply strictly with above about the chinrest liners.
- For sanitary reasons, disinfect the chinrest with the ethanol for disinfection.

Ethanol for disinfection contains 76.9 to 81.4vol% of ethanol (C₂H₆O) at 15°C (specific gravity).

4. Storage of device

1. Points to check for long-term storage

- Turn OFF the power
- Remove the power cord from the outlet
- Place the main unit undermost
- Secure the main unit by locking the slide lock of the main unit
- Put the dust cover on the main unit

2. Notes about the storage environment

Avoid storage under the following conditions:

- Where dust accumulates
- Where water may get on the unit
- Where temperature and humidity are high
- Where sunlight directly contacts
- Unstable and high place

Always follow the environment conditions for storage below.

Environmental conditions for storage

[-10°C ; +55°C]

[10% ; 95%]



Check the items above in case that the device is not used or is stored for a long time.

If you use the device after long-term storage, operate it according to the instructions of “VI > 3 > a > Preparation for measurement”.

5. Confirmation of measurement accuracy

It is extremely important to check operation and accuracy of the device with the supplied model eye.

We recommend checking its accuracy periodically.


If the measurement result of the model eye is within the tolerance listed below, the measurement is considered as reliable and accurate. If the result exceeds the tolerance, contact your dealer immediately.

Model eye data

SPH	CYL	R
Indicated value ±0.25	0±0.25	Indicated value ±0.03



The precise value of the supplied model eye is indicated on the model eye stand (VD=12).

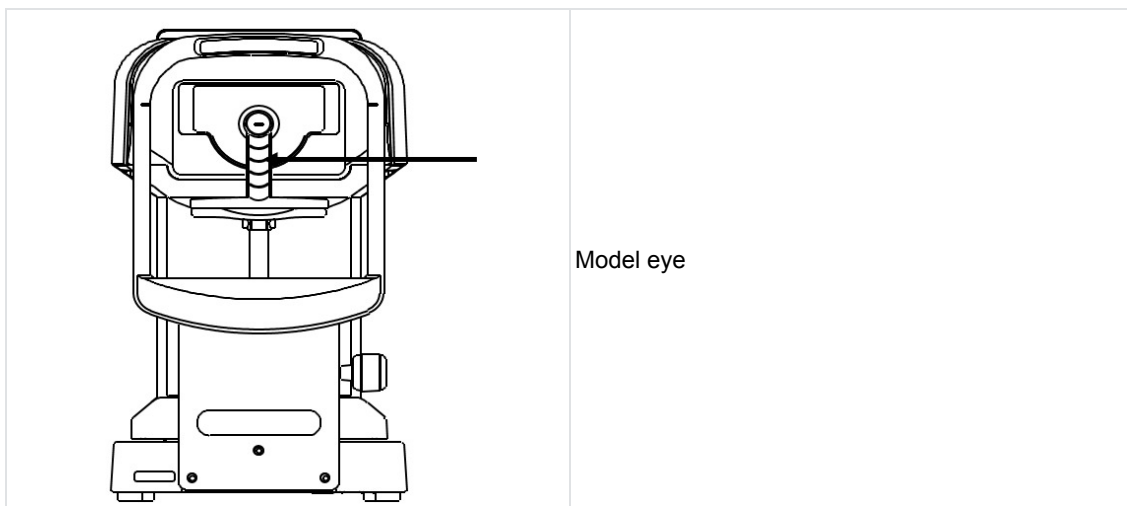

Setting of model eye



- Remove the contact lens holder and set the model eye carefully not to incline back and forth and around. It cannot take CYL value data correctly if the model eye is inclined.
- Set the model eye at the position where an alignment mark is located at the center of the reticle mark and the model eye comes into focus.
- When all of the conditions above are met, start the measurement.

6. Periodical inspection and maintenance


To prevent malfunction and accidents and maintain to performance and reliability of the product, it is recommended to request your distributor for the periodical inspection and maintenance once a year.

The periodical inspection and maintenance include inspection of the function and performance of the product, and cleaning, adjustment and replacement of consumable parts if necessary.

It is recommended for the distributors to perform the cleaning of each part, performance check and accuracy check at least once in a year.


- Cleaning of each part: exterior parts and optical system.
- Performance check: main unit and each switch.
- Accuracy check: measurement function of refractive power and corneal curvature radius.

VIII. TIPS FOR EFFECTIVE MEASUREMENT

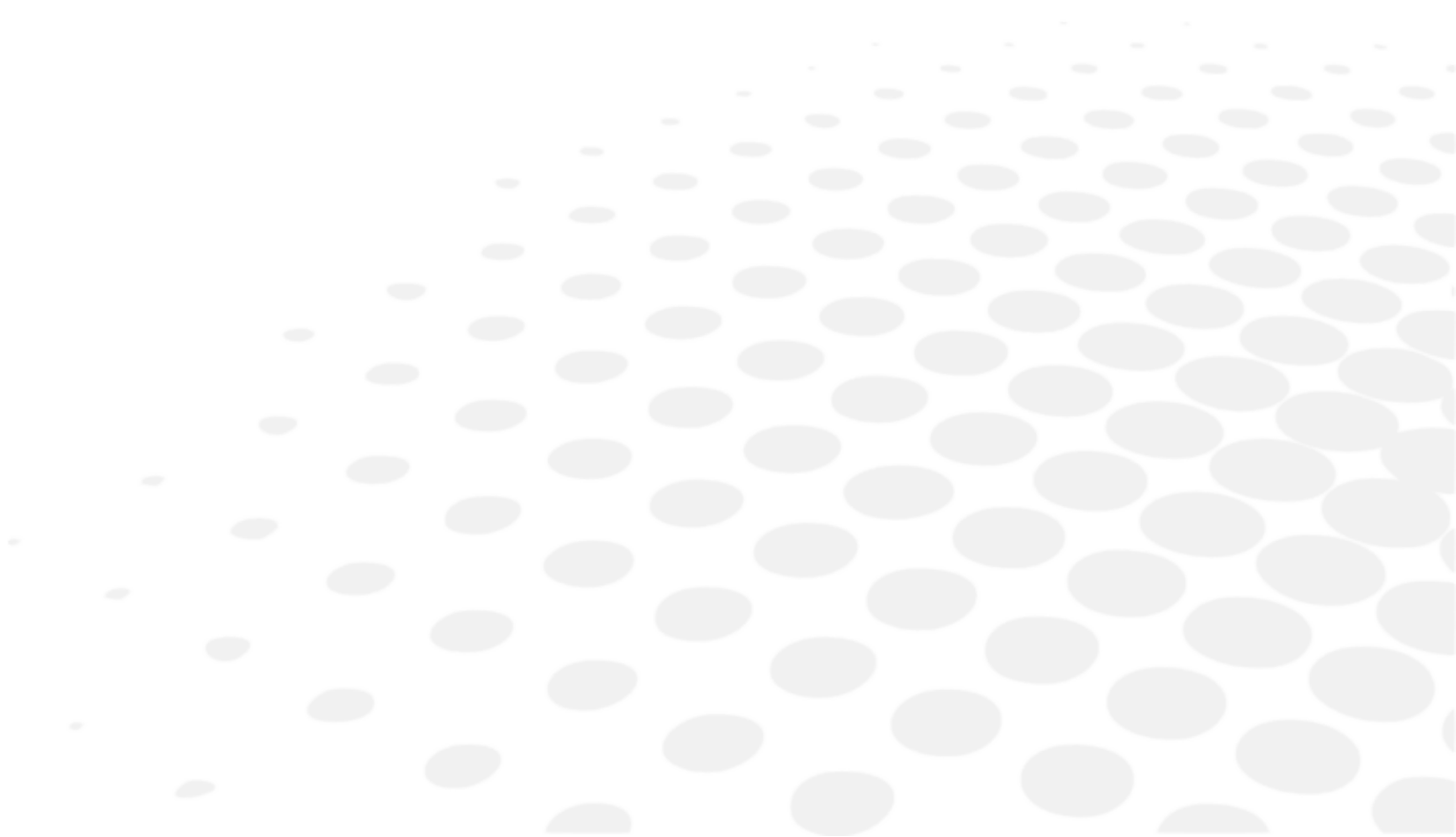

1. Do not allow external light to directly penetrate the room.
2. Fluctuation of the measurement values may occur if the examinee looks something other than the target. Urge the examinee to concentrate on the target set in front.
3. Talk to the examinee in a relaxed and friendly manner, so as to allay any fear or doubt they may have.
4. Inappropriate height of the chinrest or the chair will cause the examinee fatigue. Adjust the (optional) instrumental table to establish the most comfortable and convenient position for the examinee.
5. When the eyelash or eyelid interfere measurement, an error will occur in the measurement.
Urge the examinee to keep his/her eye open wider.
6. Tear residue or eye mucus, etc. trapped on the corneal surface may cause measurement errors. Check the surface with the LCD monitor, and if you see something moving when the examinee blinks, remove it before measurement.
7. When the pupil of the target eye is smaller than the minimum measurable pupil diameter, the device cannot take measurements correctly.
If it is difficult to take measurements because the pupil is too small, darken the surroundings (room) or the target to allow the pupil to dilate as much as possible.
8. If the examinee moves his/her head during the measurement, AXIS value will be adversely affected. Ask him/her to maintain a correct posture.

IX. ERROR DISPLAY

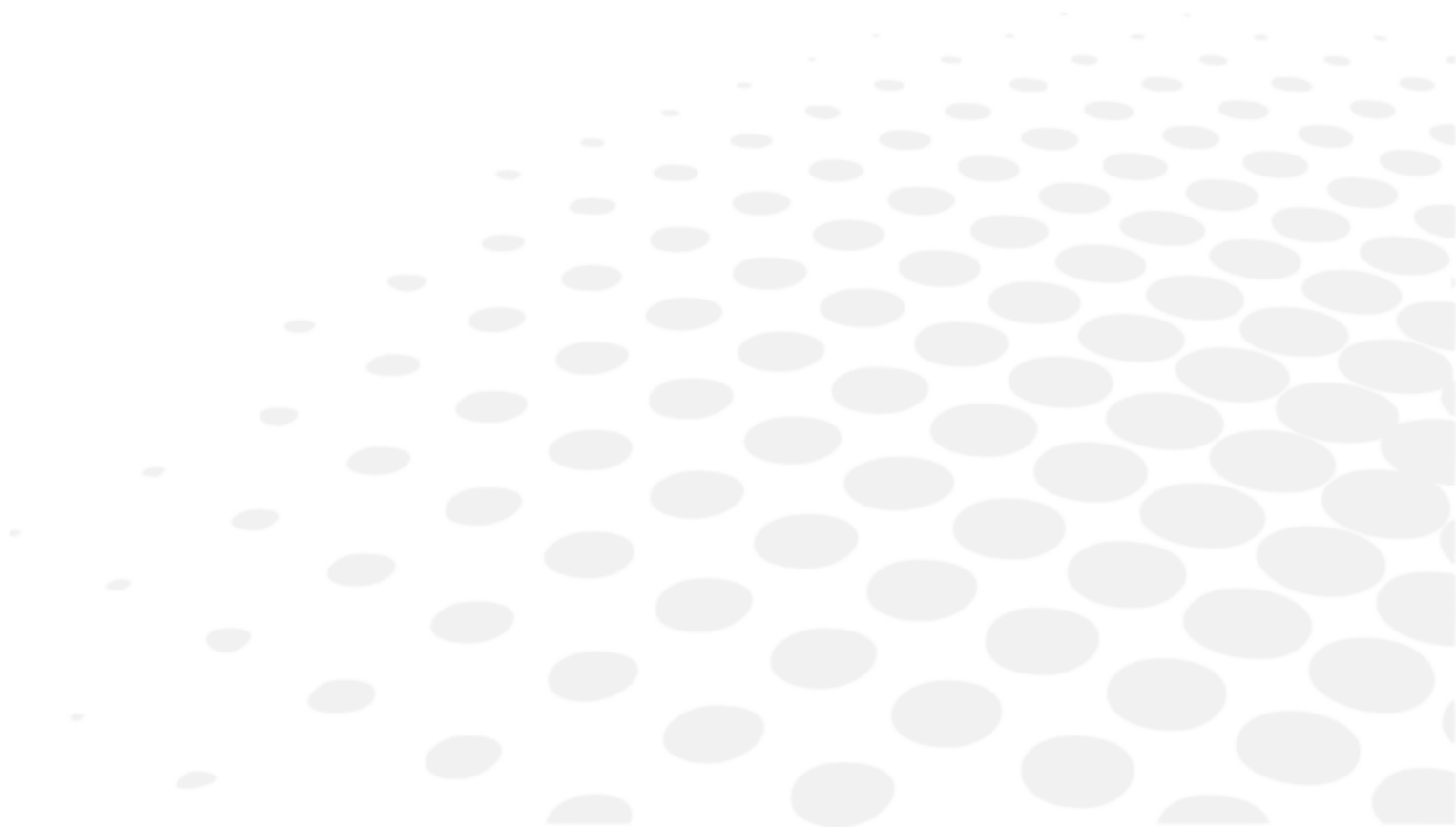


This device automatically evaluates measurement condition or result and indicates error messages if it is invalid. An error messages also appear when abnormality is detected in its operational system.

When any error messages appear, always check the system with a supplied model eye. If it appears when no abnormality in system is detected, check the measured eye for eye diseases or problems.

Message	Cause	Corrective action
RETRY	Failed to capture eye image because the examinee blinks or moves during measurement or the examined eye has eye diseases	Try alignment precisely and conduct measurement again. Consult your dealer immediately if the message appears again. Do not try to repair it by yourself.
SPH OVER	Exceeded spherical measurement range (-25 to +25D) (In case of VD=0, contact value)	/
CYL OVER	Exceeded cylindrical measurement range (0 to ±10D) (In case of VD=0, contact value)	/
ERR	Exceeded measurement value of pupil diameter (2.0 to 8.5mm)	/
Target motor fault	Detected abnormality in motor control system	Turn OFF the power and turn it back on. Consult your dealer immediately if the message appears again.
Focus motor fault		
EEPROM fault		
Printer overheated	Printer head is overheated	Turn OFF the power and turn it back on. Consult your dealer immediately if the message appears again. Do not try to repair it by yourself.
Printer cover opened	Printer cover is opened	Close the printer cover properly. Turn OFF the power and turn it back on. Consult your dealer immediately if the message appears even after closing the cover.
Paper empty	No printer paper	Set the printer paper. Refer to "VII > 1 > Reload of printer paper".

X. TROUBLESHOOTING

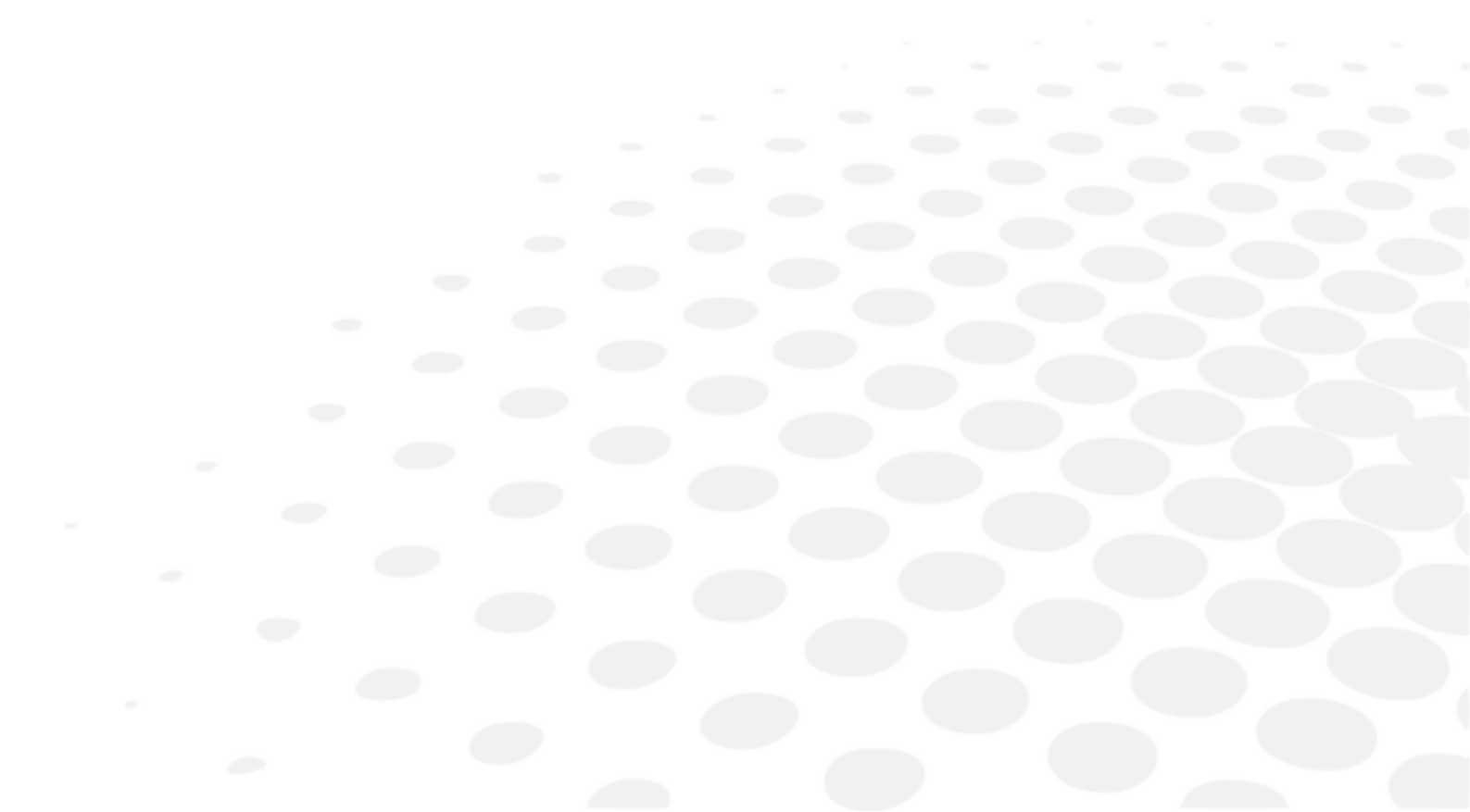


If a malfunction is found, refer to the table below to take the appropriate measures.

Symptoms	Causes and measures
The monitor and the power indicator are not turned on.	<ul style="list-style-type: none"> The power cord may not be properly connected. Make sure to connect it securely. The fuse may be blown. If so, replace it with the new one.
The fuse is blown when the power switch is turned on.	<ul style="list-style-type: none"> Contact your local distributor immediately.
The monitor display is suddenly disappeared.	<ul style="list-style-type: none"> The saving function may be activated. Press any switch to deactivate the saving function.
The moving parts such as a joystick are not moving appropriately.	<ul style="list-style-type: none"> Do not move the part forcibly. Contact your local distributor or service person.
It does not print out.	<ul style="list-style-type: none"> Check if the papers are set. Reload them if the papers are not set. The setting of Print REF/KRT may be set as OFF. Change the setting.
The printer paper comes out but not printing.	<ul style="list-style-type: none"> The printer paper may be set in a wrong direction. Set the paper properly.
The date setting is inaccurate.	<ul style="list-style-type: none"> The battery in the device may be run out. Keep the power on for 24 hours and recharge it.

Contact your local distributor immediately if the situation does not improve even after taking the measures mentioned above.

XI. SPECIFICATIONS

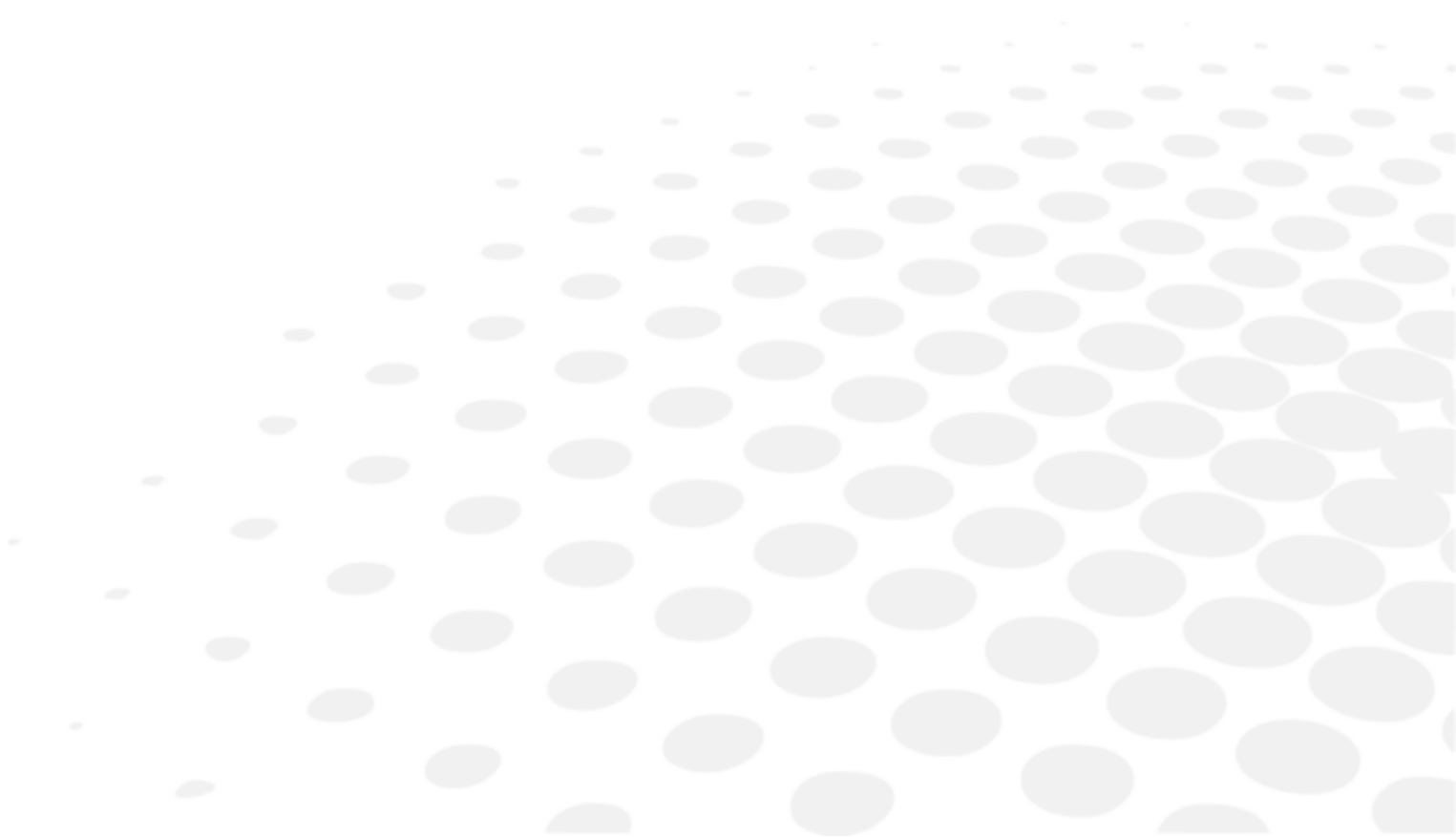


Refractive measurement range	Sphere (S): -30D to +22D	In case of VD=12 Step: 0.12/0.25D
	Cylinder (C): 0 to ±10D	Step: 0.12/0.25D
	Axis angle (A): 1 to 180°	Step: 1°
Corneal curvature radius measurement	Radius of curvature: 5.0 to 10.0 mm	Step: 0.01mm
	Corneal power: 33.75 to 67.5D	Corneal refractive n=1.3375 Step: 0.12/0.25D
	Degree of corneal astigmatism: 0 to ±10D	Step: 0.12/0.25D
	Axis angle: 1 to 180°	Step: 1°
Pupil diameter measurement	Measurement range: φ2.0 to 8.5 mm	Step: 0.1 mm
PD measurement	Measurement range: 85 mm	Step: 1 mm
Vertex distance	0, 10, 12, 13.5, 15 mm	
Minimum pupil diameter	φ2.0 mm	
Measurement time	Refractive measurement: Approx. 0.07 sec. Corneal curvature radius: Approx. 0.07 sec.	
Printer	Thermal line printer (Paper width: 58mm)	
Internal monitor	5.7 inches color LCD monitor	
Shifting range of the measurement unit	Back/forth ±22 mm Right/left ±43 mm Up/down ±17 mm	
Vertical adjustment range of the chinrest	±30 mm	
Dimensions	(W) 240 mm (D) 422 mm (H) 430 mm	
Weight	Approx. 13kg	
Output	RS-232C	
Power source	100 to 240V 50/60Hz	
Consumption	60VA	
Power saving function	OFF, 3, 5, 10 min. (switchable)	



The circuit diagram, parts lists, and description and instructions for calibration and testing are available separated from this manual.

XII. QR CODE

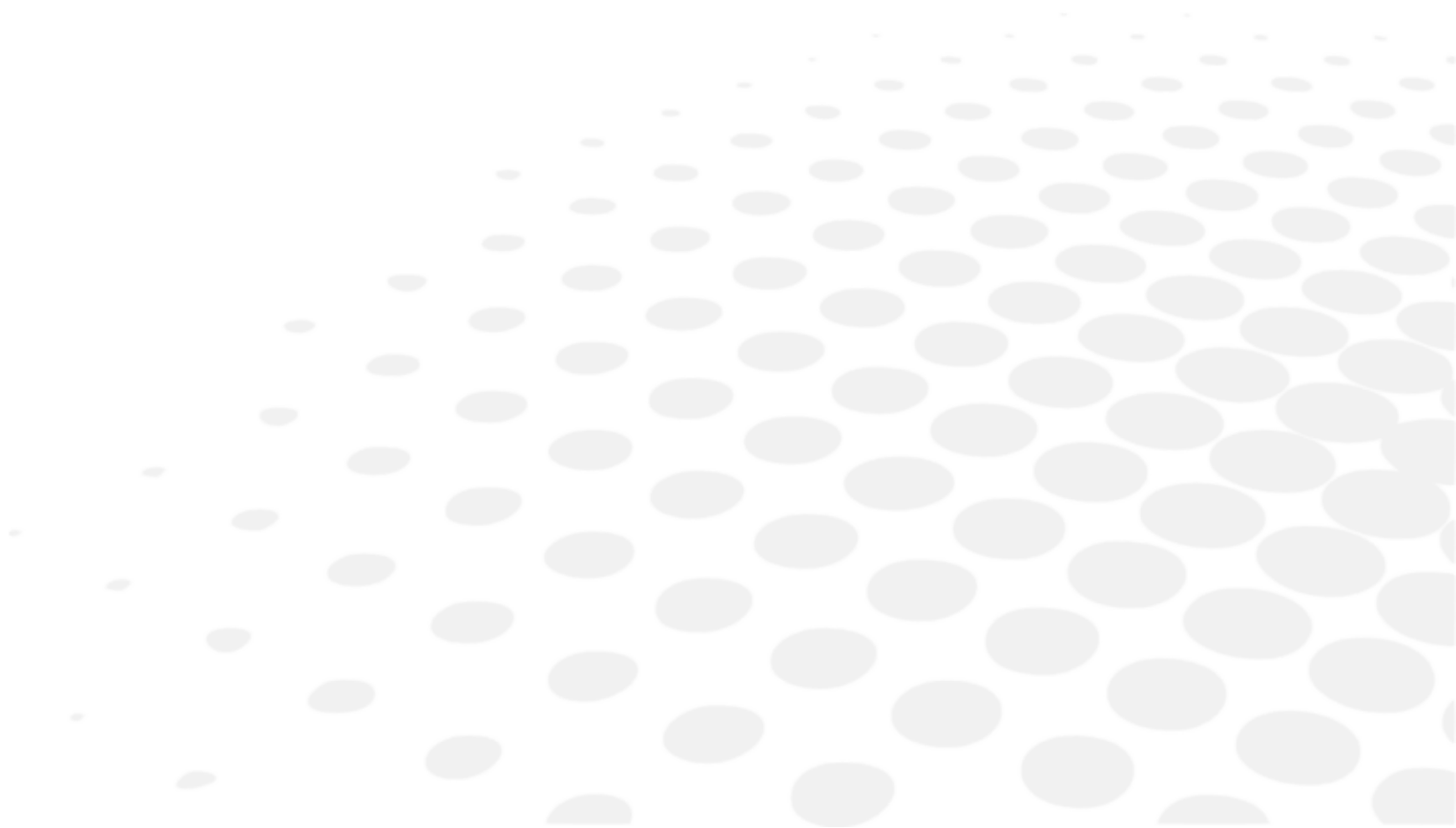

The latest version of the user manual in the appropriate language is available on a web space. Upon request, a paper version can be provided for free.

- en The complete user manual is available on a web space. To access it, please scan the QR code below using a dedicated application.
- fr Le manuel utilisateur complet est disponible sur un espace web. Pour y accéder veuillez scanner le QR code ci-dessous à l'aide d'une application dédiée.
- ar لتتمكن من الوصول إليه، يُرجى مسح رمز الاستجابة السريعة أدناه باستخدام تطبيق مخصص لذلك.
- be Поўная інструкцыя карыстальніка даступна ў інтэрнэт-прасторы. Каб атрымаць доступ, адсканіруйце QR-код ніжэй пры дапамозе спецыяльнай праграмы.
- bg Пълното ръководство за потребителя е достъпно на уеб пространство. За достъп, моля, сканирайте QR кода по-долу с помощта на специално предназначено приложение.
- cs Celá uživatelská příručka je k dispozici na webu. Pro přístup k ní oskenujte níže uvedený QR kód pomocí specializované aplikace.
- da Den komplette brugermanual findes på et websted. Du får adgang til den ved at scanne QR-koden nedenfor ved hjælp af en dertil beregnet applikation.
- de Die vollständige Bedienungsanleitung ist auf einem Speicherplatz verfügbar: Für den Zugriff darauf scannen Sie bitte untenstehenden QR-Code mittels einer dafür vorgesehenen Anwendung.
- el Το πλήρες εγχειρίδιο χρήσης διατίθεται σε έναν ιστοχώρο. Για να μεταβείτε σε αυτόν, σαρώστε τον παρακάτω κωδικό QR μέσω μιας ειδικής εφαρμογής.
- es El manual de uso completo está disponible en la web. Para acceder, escanee el código QR que se encuentra a continuación con la ayuda de una aplicación.
- et Täielik kasutusjuhend on saadaval veebis. Juurdepääsuks palun skannige allolevat QR-koodi, kasutades selleks spetsiaalset rakendust.
- fi Täydellinen käyttöohje on käytettävissä verkossa. Avaa käyttöohje skannaamalla QR-koodi asianmukaisella sovelluksella.
- hr Potpuni korisnički priručnik dostupan je na webu. Da biste mu pristupili, skenirajte QR-kod u nastavku namjenskom aplikacijom.
- hu A teljes használati útmutató megtalálható a webes felületen. A hozzáféréshez, kérjük, olvassa le a lenti QR-kódot a megfelelő alkalmazás használatával.
- id Panduan pengguna yang lengkap tersedia di web space. Untuk mengaksesnya, silakan pindai kode QR berikut dengan menggunakan aplikasi khusus.
- it Il manuale utente completo è disponibile su uno spazio Web. Per accedervi, scansionare il codice QR seguente mediante un'applicazione dedicata.
- ja ユーザーマニュアル完全版はウェブサイト内で閲覧いただけます。そちらにアクセスするには、専用アプリケーションを使用して以下のQRコードをスキャンしてください。
- ko 완전한 사용자 매뉴얼이 웹사이트에 있습니다. 전용 앱을 사용해 아래의 QR 코드를 스캔하면 접근할 수 있습니다.
- lt Išsamas naudotojo vadovas ieškokite interneto svetainėje. Kad jį atvertumėte, specialia programėlė nuskaitykite toliau pateiktą QR kodą.
- lv Pilnā lietotāja instrukcija ir pieejama tīmeklī. Lai tai piekļūtu, lūdzu, noskenējiet tālāk redzamo QR kodu, izmantojot tam paredzētu lietojumprogrammu.

ms	Manual pengguna yang lengkap boleh didapati di ruangan web. Untuk akses, sila imbas kod QR di bawah menggunakan aplikasi yang berkenaan.
nl	De volledige gebruikershandleiding is beschikbaar op een website. U kunt de handleiding bereiken door de QR-code hiernaast te scannen met een geschikte applicatie.
no	Den komplette brukerhåndboken er tilgjengelig på et webområde. For å få tilgang, må du skanne QR-koden nedenfor ved hjelp av en dedikert applikasjon.
pl	Kompletna instrukcja użytkownika jest dostępna na stronie internetowej. Aby uzyskać dostęp, zeskanuj poniższy kod QR przy użyciu dedykowanej aplikacji.
pt	O manual do utilizador completo está disponível num espaço web. Para aceder, queira digitalizar o QR code seguinte com a ajuda de uma aplicação dedicada.
pt (brazil)	O manual do usuário completo está disponível na área web do cliente. Para acessar, scanear o código QR abaixo usando a respectiva aplicação.
ro	Versiunea integrală a manualului de utilizare este disponibilă pe un site web. Pentru a-l accesa, scanați codul QR de mai jos cu ajutorul unei aplicații dedicate.
ru	Полное руководство пользователя доступно на сайте. Чтобы получить к нему доступ, сканируйте QR-код ниже с помощью специального приложения.
sk	Celý používateľský manuál je dostupný na internete. Aby ste sa k nemu dostali, naskenujte QR kód nižšie pomocou na to určenej aplikácie.
sl	Celoten uporabniški priročnik je na voljo na spletnem mestu. Za dostop do njega skenirajte spodnjo kodo QR z uporabo namenske aplikacije.
sr	Potpuno korisničko uputstvo je dostupno na vebu. Da biste mu pristupili, skenirajte QR kôd u nastavku pomoću namenske aplikacije.
sv	Den fullständiga handboken finns på en plats på Internet. Skanna QR-koden nedan med en lämplig app för att få åtkomst till den.
th	มีคู่มือผู้ใช้ฉบับสมบูรณ์ให้ที่เว็บไซต์ เพื่อเข้าถึงข้อมูล กรุณาสแกนรหัส QR ด้านล่างนี้โดยใช้แอปพลิเคชันเฉพาะงาน.
tr	Kullanma kılavuzunun tamamı internette bulunmaktadır. Kılavuza erişmek için, bu amaca yönelik bir uygulamaya kullanılarak aşağıdaki QR kodunu taratın.
uk	Повний посібник користувача доступний на сайті. Щоб отримати до нього доступ, скануйте QR-код нижче за допомогою спеціального додатку.
vi	Cẩm nang hướng dẫn sử dụng hoàn chỉnh hiện có trên không gian web. Để truy cập, vui lòng quét mã QR bên dưới sử dụng ứng dụng chuyên dụng.
zh	操作手册全文可在一个网络空间内查询。如要访问该空间，请使用一个专门的应用软件扫描QR条码。





Essilor International
147, rue de Paris – 94220 Charenton-le-Pont France
www.essilor.com

